ABC Breast Care

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INSTRUCTIONS FOR USE - ENGLISH

Thank you for choosing ABC Breast Care. ABC products are developed and manufactured with great care and knowledge of the needs of breast-operated women.

This instruction manual applies to:

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Please note that not all ABC products are available in all countries.

Scope of delivery:

- ABC Swim Form
- Waterproof bag for travel and storage
- Instruction manual

1. Intended Use/Indication

- ABC Swim Forms are breast-shaped transparent pads to compensate for removed/missing breast tissue.
- ABC Swim Forms are intended for external use only.
- ABC Swim Forms are intended for swimming (saltwater friendly), workouts & various water activities.
- 980 ABC Active Swim Form: Ribbed, hollow back to allow for optimal air and water flow.
- User group: Patients between 12 and 99+ years of age.
- **Please note:** Your ABC Swim Form is not a long-term substitute for wearing a regular silicone breast form.

2. Risks/Contraindications

- There are no known risks or contraindications to be expected when wearing ABC Swim Forms under normal conditions and in accordance with the care instructions. No safety issues or performance impairments are expected during normal use of the ABC Swim Form.
- ABC Swim Forms should not be used in contact with open or broken skin.

- Do not wear your ABC Swim Form until your surgery site is fully healed and radiotherapy is complete.
- If skin redness or rashes occur, please stop using the product and contact your doctor and retailer.

3. Material

- Polyurethane film
- Colour free/transparent Silicone gel based on Polydimethylsiloxane
- ABC Swim Forms are carefully checked for quality and safety.

4. Instructions for use

- ABC Swim Forms are intended to be used in the sewn-in textile pockets of a swimsuit.
- The symmetric design can be fit on either left or right.
- Avoid bringing your ABC Swim Form in direct contact with perfume, hot surface, abrasive cleaners, sunscreens, and insect repellents.
- Avoid sharp, pointed objects near your ABC Swim Form. Pins, fingernails, pet claws, scissors, etc., all cause permanent damage.
- Avoid stretching, twisting, rubbing, or using excessive force when handling your ABC Swim Form.

5. Care instructions

- Clean your ABC Swim Form after use with a mild soap and dry it with a soft towel.
- Chlorine or salt water will not harm your ABC Swim Form.
- The waterproof bag included in the scope of delivery is ideal for short-term transport of your wet swim form and swim suit.
- For longer storage in the bag, make sure that your swim form and the inside of the bag are completely dry to prevent mould growth.

6. Warranty

- The statutory warranty applies.
- Should your ABC Swim Form become defective, please contact your specialist medical retailer. In the case of a clear production or material defect, ABC will be happy to replace the product within 1 year from the date of purchase.
- Discoloration of the ABC Swim Form may occur. Such discoloration will not affect the integrity or quality of the product.

7. Incident Reporting

Only serious incidents relating to the product, which lead to a serious deterioration of the state of health or death, must be reported to the manufacturer or competent authority of the EU Member State. Serious incidents are described in Article 2 (65) of Regulation (EU) 2017/745 (MDR).

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Store dry

Disposal

Household disposal – in accordance with national legal regulations

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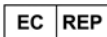
ABC is an ISO 9001 and ISO 13485
registered company.

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ABC Breast Care GmbH