
ARION Easy-Slide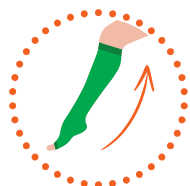


EN	Donning aid for medical compression stockings with an open toe	Instructions for use	2
DE	Anziehhilfe für medizinische Kompressionsstrümpfe mit offener Fußspitze	Gebrauchsanleitung	4
NL	Aantrekhulp voor medische compressiekousen met open teenstuk	Gebruiksaanwijzing	6
FR	Aide à l'enfilage de bas de compression médicale à pointe ouverte	Mode d'emploi	8
ES	Calzador de medias de compresión médica con puntera abierta	Instrucciones de uso	10
PT	Auxiliar para calçar meias de compressão médica de ponteira aberta	Instruções de utilização	12
IT	Dispositivo per mettere le calze compressive medicali con punta aperta	Istruzioni per l'uso	14
DA	Hjælp til påtagning af kompressionsstrømper med åbne tæer	Brugsanvisning	16
SV	Påklädningshjälpmedel för medicinska kompressionsstrumpor med öppen tå	Bruksanvisning	18
NO	Påføringshjelp for medisinske kompresjonsstrømper med åpen tå	Bruksanvisning	20
FI	Pukemisen apuväline lääkinällisille kompressio- sukille, joissa on avoin varvasosa	Käyttöohjeet	22

Arion Easy-Slide

Donning aid for medical compression stockings with an open toe

Important information before you start using the product:

- A.** Useful tips:
 - Wear special non-slip gloves when using the product. This gives you maximum grip on the stocking and reduces the physical strength needed.
 - In case the compression garment has a silicon border, make sure to turn it outwards.
- B.** Make sure the product is ready for use:
 - Fold back the extra fabric into the „green bag“
 - Align the fold line with the top of the orange labels
- C.** If you have difficulty to reach your toes, please place your foot on a low chair to avoid bending too much.

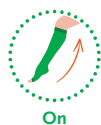On

Donning instructions:

1. Slide the Easy-Slide over the foot
2. Slide the stocking over the Easy-Slide
3. Draw the stocking over the heel
4. Place one hand firmly on the heel and with the other hand pull the loop of the Easy-Slide
5. Pull the Easy-Slide out in small movements

SIZING

REF	Size	Size Shoe EU	Shoe size UK		Shoe size US	
			women	men	women	men
Eo 1500	Extra small	≤ 33	≤ 1	≤ 2	≤ 3 ^{1/2}	≤ 2 ^{1/2}
Eo 1501	Small	34 - 36	1 ^{1/2} - 3 ^{1/2}	2 ^{1/2} - 4	4 - 6	3 - 4 ^{1/2}
Eo 1502	Medium	37 - 38 ^{1/2}	4 - 5 ^{1/2}	4 ^{1/2} - 6	6 ^{1/2} - 8	5 - 6 ^{1/2}
Eo 1503	Large	39 - 41	6 - 7	6 ^{1/2} - 7 ^{1/2}	8 ^{1/2} - 9 ^{1/2}	7 - 8
Eo 1504	Extra large	≥ 42	≥ 7 ^{1/2}	≥ 8	≥ 10	≥ 8 ^{1/2}

Easy-Slide from Arion Slide Solutions minimizes friction between compression garment and skin thanks to the product's unique sliding properties provided by the special material it is made of and by its coating. The friction reduction is particularly helpful to slide the garment over the heel area, which is the broadest part of the foot.

EASY, SAFE AND SIMPLE SOLUTION

- Reduces physical strain for both compression wearers and caregivers
- May contribute to a longer life for the garment and to maintaining the original compression longer by preventing damage from fingers or nails and avoiding overstretching and distortion
- Promotes independence of compression wearers and supports therapy compliance

COMPOSITION

- Product: Polyamide 93%, Polyurethane 3%, Silicone 2%, Polyester 2%
- Package: Chromo carton GC1 (PAP21) 

CONTRAINDICATIONS

- Do not use the product if you are allergic to one of its component (see composition).
- Open wounds should be covered with an appropriate dressing before using the product.

⚠ WARNINGS

- **Suffocation hazard:** Keep the product away from babies, children, and pets.
- **Slipping hazard:** Do not step on the product. Highly slippery material.
- **For external use only /** Intended for skin contact. Seek medical attention if irritation develops.

IMPORTANT

-  Please **read the instructions for use carefully**. If you have any questions, please contact your reseller.
- Please **keep the package and the instructions for use during the time of use of the product**.
- All **serious incidents** that occur when using the product **should be reported** to the manufacturer and the competent supervisory authority.
- When using the product it is important to **use the right product size** for the user. Using an incorrect size will prevent the product from working properly, and it may become damaged or wear out too quickly. The correct size is determined by your shoe size.
- **Use the product on a single person.** If the product needs to be used on more than one person, disinfect the product before with a 70% ethanol solution.

STORAGE

Keep away from direct sunlight or other heat sources. 

CARE INSTRUCTIONS

- Wash by hand, at max. 30°C/86°F.     
- Do not bleach.
- Do not tumble dry. Dry the product only with a towel.
- Do not iron.
- Do not dry clean.
- Disinfection possible with a 70% ethanol solution.

GUARANTEE

This product is guaranteed for a period of six months from the date of purchase against defects resulting from faulty material or construction. Within the guarantee period, the product will be exchanged free of charge in the case of manufacturing and/or material defects. Any exchange under the guarantee will only be carried out if it can be demonstrated by means of a purchase receipt that the date on which the complaint was lodged falls within the guarantee period. The guarantee does not apply if the defect is due to improper use of the product or use on several patients. To use this service you need to contact your supplier. You are asked to return the product washed and to follow the care instructions.

RECOMMENDATION FOR USE

 The manufacturer recommends replacing the product six months after the first use if the product is used regularly.

You can write the date of first use on the back of this document.

Watch our **instruction video** here: uqr.to/EasyEN or scan this QR code:

More information on www.slide-solutions.com

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