BORT X-Walker AIR kurz

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




Med
Mitten im
gesunden
Leben

Illustrationen
Figures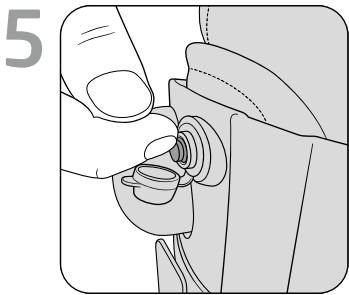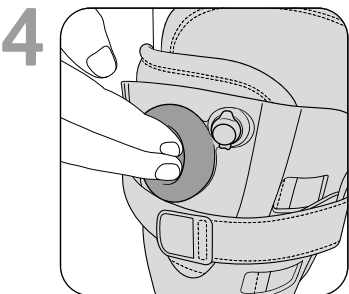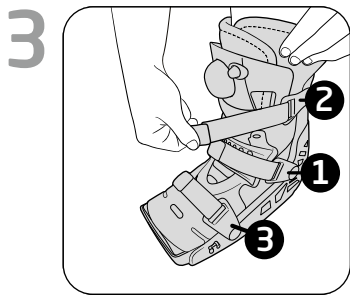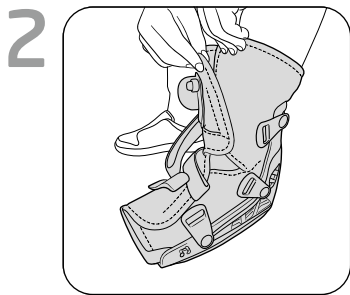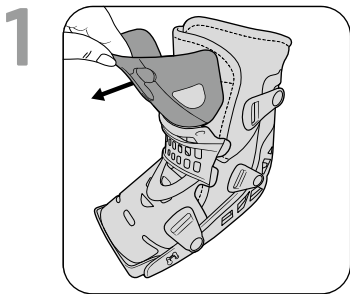

Sprachen Languages

DE	Deutsch	Gebrauchsanweisung	04
EN	English	Instructions for use	08
FR	Français	Mode d'emploi	11
ES	Español	Instrucciones de uso	15
IT	Italiano	Istruzioni per l'uso	19
NL	Nederlands	Gebruiksaanwijzing	23
CS	Čeština	Návod k použití	27
ET	Eesti	Kasutusjuhend	30
PL	Polski	Instrukcja użytkowania	33
RO	Română	Instrucțiuni de utilizare	37

 PDF: ga.bort.com

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Many thanks for placing your trust in a medical device from BORT GmbH. Please read the existing instructions for use carefully. If you have any questions, please refer to your physician or the specialist retailer from whom you purchased this medical device.

Intended purpose

This medical device is a lower leg foot brace for complete immobilisation of the lower leg and foot region in a specified position.

Indications

Pre-operative or post-operative/post-traumatic immobilisation of the forefoot and metatarsus region, metatarsal fracture, soft tissue injuries, distortions.

Contraindications

Thrombosis risk, extreme varicosis, peripheral arterial occlusive disease (PAOD), lymph drainage disorders, also unclear soft tissue swellings distal to the aid positioned, sensory and circulatory disorders of the region of the body treated, skin diseases in the part of the body treated.

Application risks/Important notes ⚠

This medical device is a prescribable product. Discuss the use and duration with your treating physician. The expert staff from whom you have received the medical device will select the appropriate size and instruct you regarding its use.

- remove the medical device prior to radiological examinations
- the Walker is appropriate for a maximum body weight of 130 kg
- if wearing at night is prescribed, avoid negative influence on the circulatory system
- in case of numbness, loosen the medical device and remove it if necessary
- in case of persistent complaints, consult the physician or specialist retailer
- use the medical device in accordance with therapeutic needs
- only use other products simultaneously after consultation with your physician
- do not make any changes to the product
- do not wear it on open wounds
- do not use in case of intolerance of one of the materials used
- no re-use. This medical aid is intended for treating one patient only
- whilst wearing the brace, please neither use any creams nor ointments on or around it as they can destroy the material

Fitting instructions

When wearing the brace, please wear one of the two understockings provided on the foot affected. Pay attention to it sitting crease-free.

- 1** Then, open the circumferential Velcro fasteners and remove the front cover. In order to make it easier to put on later, we recommend twisting the Velcro straps downwards, thus easing application of the brace.

2 Open the padding liner and place the foot and lower leg in the Walker. Pay attention to good heel positioning, it should be placed securely in the very back of the heel rest. The padding liner is then closed on the foot and in the lower leg region using the Velcro fastener, pay attention to it sitting crease-free.

Information for the technician:

Setting the foot length:

If required, the foot length can be adapted to the patient's circumstances by moving the front binding. For this purpose, the front binding is raised and pulled upwards gently to disengage it. Following this, it can be moved and be locked in the desired position by pressing it downwards gently.

3 Then, adapt the grey front cover again centrally on the padding liner. The pump is attached to it using Velcro and should be positioned in the central recess in the front fitting. If required, you can adapt the position of the pump on the padding liner.

Close the four circumferential Velcro fasteners in the order specified below:

1. Start with the fastener on the ankle joint.
2. Close the lower leg fastener.
3. Close the strap in the metatarsus/toe region.

If necessary, re-tighten again.

4 To pump the air into the liner and thus generate stabilisation, then activate the pump several times until you sense a surface stabilisation in the lower leg and foot region. If necessary, re-tighten the fastening straps again.

Scope of delivery:

BORT X-Walker AIR short incl.:

- Liner X-Walker AIR short
- Understocking Walker (pair)

Accessories available:

REF 100 370 BORT Understocking Walker

REF 100 380 BORT Protective cover against wetness

REF 100 390 BORT Protective cover for night-time

Removal

5 To remove the aid, remove the protective cap from the side of the pump and press the bronze-coloured button downwards. The air then flows out of the padding liner audibly. Then, open the harnesses, remove the front cover and open the liner to remove the brace from the leg. Close all the Velcro fasteners in order to avoid damaging other items, e.g. to further clothing.

Material composition

Brace rest:	100 % polypropylene (PP)
Liner (cover):	35 % polyester (PES), 35 % cotton (CO), 30 % pure ester (PUR)
Fasteners:	100 % polyamide (PA)
Understockings:	88,5 % cotton (CO), 9,0 % polyamide (PA), 2,5 % elastane (EL)

Cleaning information

 Hand-washing 30 °C  Do not bleach (only wash with colour or mild detergents)

 Do not dry in a tumble dryer  Do not iron  Do not clean chemically

Clean the brace rest manually with lukewarm water, wash the liner cover manually with mild detergents at a maximum of 30° C. Air dry whilst avoiding the impact of heat.

Guarantee

The legal regulations of the country in which you acquired the product apply to the purchased product. Please contact your specialist retailer if you suspect a warranty claim. Please clean the product before submitting a warranty claim. If the enclosed instructions for use have not been properly observed, the warranty may be impaired or cancelled. The warranty does not cover use of the product inappropriate for the indication, non-observance of application risks, instructions and unauthorised modifications to the product.

Useful life/Lifetime of the product

The lifetime of the medical device is determined by its natural wear and tear if treated appropriately and as recommended.

Duty of notification

If a serious worsening of the state of health occurs when using the medical device, you are obliged to notify the specialist dealer or us as the manufacturer and the Medicines & Healthcare products Regulatory Agency (MHRA).

You can find our contact information in these instructions for use. You can find the contact information for the national authority responsible for your country under the following link: www.bort.com/md-eu-contact.

Disposal

Upon the termination of use, the product must be disposed of in accordance with the corresponding local requirements.

Declaration of conformity

We confirm that this device conforms with the requirements of REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL. You can find the current declaration of conformity under the following link: www.bort.com/conformity

Status: 10.2025

 Medical device |  Single patient – multiple use

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