POWERSTROLL

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Table of contents

Preface	11
Declaration of conformity	11
Service life	11
Disposal	11
Identification plate	11
Reuse	11
Warranty	11
Parts of the product	12
Intended purpose	12
Indication Contraindication	12
Scope of delivery	12
Safety instructions	13
Introduction	14
Electromagnetic compatibility (EMC)	14
Assembly	14
Use	15
Battery	16
Disassembly and transport	17
Cleaning Maintenance	17
Troubleshooting	17
Warranty conditions	17
Technical data	17

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

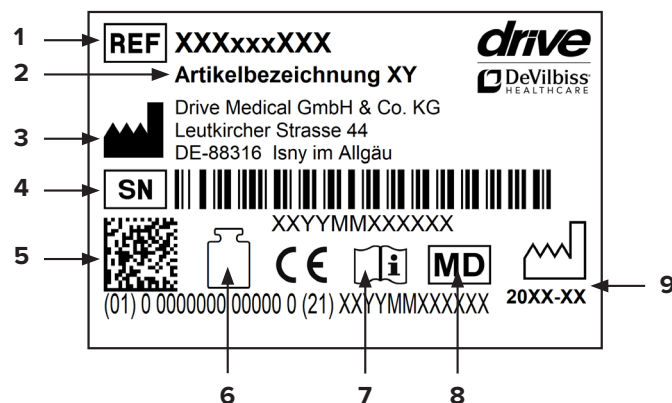1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at www.drivedevilbiss.de

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others):
Straps and tires

WARRANTY BATTERY

6 Months

Parts of the product

PowerStroll

1. Display
2. Throttle lever
3. Battery
4. Power unit
5. Relief strap
6. Adapter for holder

Intended purpose

Wheelchair pushing aids for third-party use are controlled by the assistant. Controlling the speed and direction is usually through a control unit mounted on the push handles. These devices enable assistants (wheelchair pushing aids for third-party use) to fit manual wheelchairs with an electric drive, without having to make any significant changes to the wheelchair. This means that the benefits of a manual wheelchair are retained.

Indication | Contraindication

- Limitations in the control of an electric power unit
- To ensure mobility outdoors (getting to local areas and performing everyday activities) by an assistant/caregiver who does not have sufficient strength of his/her own.

The assistant/caregiver who operates the wheelchair

pushing device must be instructed in the proper operation of the power unit (including connection and disconnection) and be able to operate the device properly.

Scope of delivery

1 x	Manual control
1 x	Relief strap
1 x	Battery
1 x	Power unit including fastening materials
1 x	Telescopic mounting bar
1 x	Telescopic mounting bar up to 70 cm (only for PowerStroll HD)
2 x	Adapter for holder (1x preassembled)
1 x	Carrying bag
1 x	Instructions for use

Safety instructions

General

- For reasons of safety, the pushing aid should be operated only by persons who have been trained in its handling and who are physically and mentally capable of operating the PowerStroll in all operating situations.
- The PowerStroll may under no circumstances be used by the wheelchair user alone.
- During assembly, it must be ensured that the wheels of the PowerStroll are as close as possible to the center of the 24" wheels of the wheelchair. In some wheelchair models, the pushing aid can also be mounted in front of the cross-brace of the wheelchair. This allows more legroom for the assistant.
- We recommend always mounting anti-tilt aids, otherwise there is a risk of tilting and injury when going up inclines.
- The PowerStroll should be set to a speed which is comfortable for both the assistant as well as the wheelchair user.
- If the battery is connected to or disconnected from the PowerStroll or the charging unit, it should be ensured that both the charging unit as well as the manual control of the PowerStroll are switched off.
- The PowerStroll must be secured during transport. It is also recommended to unplug the battery.
- The battery may be charged only with the charging unit supplied with the PowerStroll. The use of another charging unit can lead to damage or injuries.
- The PowerStroll may be influenced by certain electromagnetic waves. Close proximity to mobile telephones, radio transmitters, and radio stations of emergency services should be avoided. Influences by other household devices are not known. In the event of unintentional driving behavior, switch off the PowerStroll and contact your dealer.
- Do not use the PowerStroll in snow or heavy rain. The PowerStroll could be damaged as a result. The PowerStroll should be protected against moisture or long-term direct sun exposure.
- Do not drive your PowerStroll over a deep, soft surface (such as soft sand, thick grass, loose gravel).
- NEVER drive your PowerStroll through deeper water and never clean your PowerStroll with a high-pressure cleaner.
- The control system must always be switched off when not in use.
- To prevent unauthorized use, simply remove the control panel after unplugging when not in use and store it in a dry, safe place until the next use. A click closure allows you to remove or replace the control panel without needing any tools.

Battery

- Do not drop, strike, crush, or otherwise abuse the battery since the corrosive battery cell contents can be exposed as a result.
- If the battery leaks, do not allow the liquid to come into contact with skin or eyes. In the case of contact, wash the affected area with plenty of water and seek medical advice.
- Do not expose the battery to fire or extreme heat. Do not burn. Do not expose the battery to intense heat since this can cause an explosion. Avoid storage in direct sunlight.
- Do not expose the battery to water, rain, or any type of moisture.
- Do not crush, disassemble, pierce, or short-circuit the connection terminals.
- Do not open, disassemble, or try to repair the battery; it does not contain any parts which can be repaired.
- Do not arbitrarily store the batteries in a container in which they can short-circuit each other or be short-circuited by other metal objects.
- Keep batteries out of the reach of children.
- Keep batteries clean and dry.
- Use the battery only for the application for which it is intended.
- Regularly check the connection cable, connection heads, and power supply for a secure fit, damage, or signs of wear. Do not use if damaged.
- Fully charge the battery prior to initial use. Maximum time between charges = 6 months.
- Recommendation: Store the battery at below 25°C, with low air humidity, and in a dust-free and non-corrosive gas atmosphere. Whenever possible, store batteries fully charged.
- This device contains electrical and/or electronic components. Be aware of local regulatory and recycling regulations for disposing of the device components.
- Dispose of the battery properly.
- Every serious incident which occurs in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.

Introduction

- The PowerStroll is a pushing aid for manual wheelchairs. It is designed for use indoors and outdoors and exclusively for transport by an assistant. The pushing aid is intended to make pushing easier for the assistant in difficult situations, such as in the case of steep uphill inclines.
- The PowerStroll has a few special features which facilitate the installation and operation of the device.
- The PowerStroll pushing aid is for the operation of a manual wheelchair by an assistant for a total weight of wheelchair and wheelchair user of up to 135 kg.
- The PowerStroll contains a 24 V 12Ah battery and has a range of up to 16 kilometers on one charge.
- Since the design is subject to changes, some of the illustrations and photos in the manual may not match the pushing aid you purchased. The manufacturer reserves the right to make design changes.


Electromagnetic compatibility (EMC)

This product was tested for electromagnetic compatibility (EMC) according to IEC 60601-1-2 and ISO 7176-21.

Assembly

Manual control

Loosen both screws on the molded handles (**photo 1**). Put the holders on the push handle and secure with the screws (**photo 2+3**). Secure the display to the holder (**photo 4**).

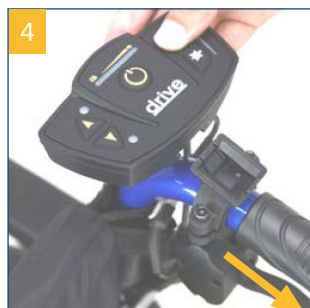
Note: The display can – facing inwards or outwards – be attached to the left or right push handle, depending on what is more comfortable for the user.

Power unit

Push the telescopic bar on the front end through the holder on the power unit. Loosen the star screw on the telescopic bar and adjust to the correct length. Adjust the telescopic bar to the width of the holders and push forward until both sides have clicked into place.

Then align the PowerStroll in the center and secure with the wing screws. When the PowerStroll is attached to the wheelchair, it should be in a horizontal position. If this is not the case, orient the power unit using the height adjustment on the PowerStroll (**photo 6**).

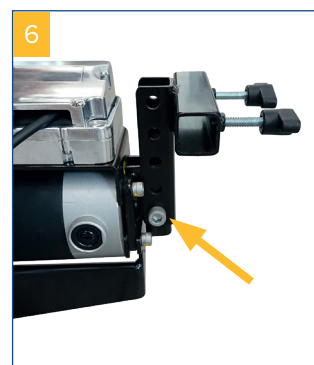Mount both holders on the lower horizontal tubing of the wheelchair in front of the scissor lift mechanism (**photo 7**). If the PowerStroll can be assembled in front of the scissor lift mechanism, the preassembled adapter may have to be replaced with the accompanying extended adapter.

Tighten the screws so that the holders sit securely and do not rotate.

Holder

Telescopic mounting bar (**photo 8**):

The PowerStroll is suitable for most wheelchairs with a seat width of 40 cm – 50 cm.



The PowerStroll can be used with the HD telescopic bar (article number: 960000043 - **photo 9**) also for seat widths from 38 cm to approx. 70 cm.


Battery

Insert the battery into the battery compartment, as pictured. Then push the battery forward until it visibly snaps into place (**photo 10+11**).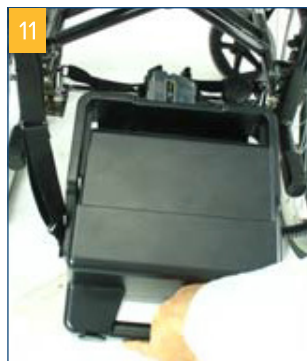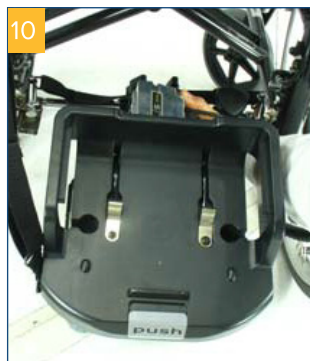

Connection

Connect the spiral cable of the power unit to the display (**photo 12**) and place the blue loop of the relief strap around the push handle opposite the manual control.

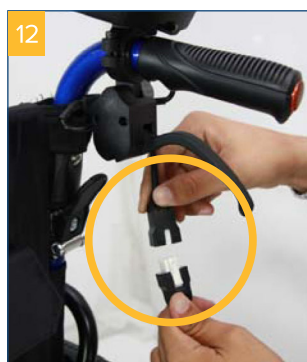Use

Manual control

To switch the PowerStroll on or off, push the on or off switch (**5**). Please be aware that it takes a moment after switching on the PowerStroll until it is ready for use. Wait 2 seconds before operating the throttle lever (**photo 13**).

The direction of travel can be adjusted by pushing the arrow buttons (**1+6**).

The max. speed can be preset by pressing the rabbit (fast) and turtle (slow) buttons (**3+4**).

The battery display (**2**) shows how much capacity is still available. If the battery is fully charged, all LEDs in the battery display are lit; as the capacity of the battery decreases, one LED goes out gradually from right to left. If there is a problem with the PowerStroll, the display flashes. This may mean:

- Single flash – battery weak. Charge battery.
- 7 flashes – throttle lever.
- Release the throttle lever and restart the PowerStroll. In the case of another flash sequence, restart the PowerStroll.

If this does not correct the problem, contact your Drive Medical dealer.

Maximum speed

Always set a maximum speed at which the assistant can maintain a comfortable pace.

Throttle lever

To start moving with the PowerStroll pushing aid, squeeze the throttle lever (**photo 14**). The harder the throttle lever is squeezed, the faster the pushing aid moves. To stop the PowerStroll, release the throttle lever. The direction of travel is adjusted using the forwards or backwards button (**photo 13**).

Relief strap

The PowerStroll pushing aid is supplied with a relief strap so that the wheelchair can also be moved without the PowerStroll. Lift the device with the red pull cord and place the loop around the push handle (**photo 15**).

Battery

Follow the instructions below step by step:

Charging the battery

1. Switch off the pushing aid.
2. Remove the battery.
3. Insert the round plug of the charging unit into the charging socket of the battery.
4. Insert the power cord of the charging unit into the outlet.
5. The LED of the charging unit is lit and red when the battery is being charged. The charge time is about 8 hours, depending on the charge status of the battery. For optimal performance, a charge time of 12 hours is recommended.
6. The LED changes from red to green when the battery is fully charged.
7. Switch off the charging unit, unplug the power cord, and pull the round plug out of the battery pack.

Battery care

1. Unplug the charging cable only after the battery has been fully charged. The service life of the battery shortens drastically if the battery is repeatedly used when not fully charged. Therefore ensure that the battery is fully charged each time.
2. Always charge the battery until the LED changes from red to green. NEVER interrupt the charging process before it has been completed.
3. When the battery is fully charged, the charging unit starts maintenance charging to ensure an optimal range.
4. If the pushing aid is not used for a longer period of time, charge the battery at least once per month in order to keep the battery in a fully charged state at all times.
5. The environmental temperature has an effect on the charge time and capacity of the battery. The range is decreased at cold temperatures and the charge time is increased.
6. Do not leave the charging plug inserted in the battery if the charging unit is unplugged since this drains the battery and the range is reduced.

7. Only use the charging unit supplied and fully charge the battery each time. The battery can become damaged if a charging unit which does not meet the specifications is used.
8. Do not disassemble or modify the charging unit.
9. Charge the battery in a well-ventilated room where the battery is not exposed to direct sunlight. Do not charge the battery in a damp or wet environment.
10. Do not charge the battery at temperatures below -10°C or over +50°C since such temperatures can impair the function of the charging unit and the battery can become damaged.
11. During charging, keep the battery away from flammable objects and open flames, since the battery could ignite or burst.
12. Do not smoke during charging since the battery can release hydrogen gas. Always charge the battery in a well-ventilated room.
13. During charging, never disconnect or connect the plug or the cable with wet hands; if they are wet, this can lead to an electrical shock.
14. The battery warranty is good for six months. This warranty applies only to manufacturing errors and not to errors which can be attributed to incorrect charging not in accordance with the instructions.


Charging unit

The charging unit supplied has an LED which changes color depending on the charge status:

- If the LED is lit and RED, the charging unit is switched on and charging.
- If the LED is lit and GREEN, the charging unit is switched on, the majority of the charging has been completed, and the battery can be used. However, it is recommended to leave the charging unit to continue charging for several more hours after the green LED appears. The battery performance is maintained or increases as a result.
- The LED is also lit and GREEN if the charging unit is switched on but not connected to the battery.

Disassembly and transport

1. Switch off the PowerStroll and unplug the cable at the control panel.
2. Remove the relief strap from the push handle.
3. To release the battery, press the gray PUSH button on the power unit (**photo 16**). Then pull out the battery.
4. Remove the power unit. To do this, pull the black band between the holders upwards and at the same time, take the power unit out of the holder from the back (see image below).
5. Stow all parts in the carrying bag provided.

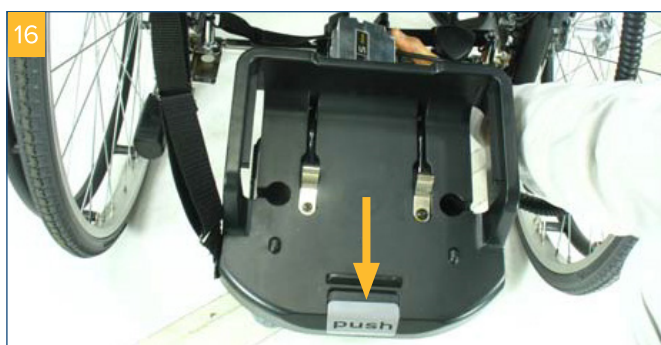Note

The battery supplied with the PowerStroll meets IATA special provision A67 and is approved for transport in an airplane. Nonetheless, if you have any questions in this regard, consult your airline prior to travel.

Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product must not be continuously exposed to temperatures over 60°C.
- In the event of defects, contact your specialist dealer from whom you purchased this product.
- Do not use any petroleum ether, solutions or evaporating solutions since these could damage plastic parts. Do not use any wax.
- Ensure that the charging unit was unplugged and that the PowerStroll is switched off before you clean it.

- Check the condition of the tires regularly. They are "fail-safe," meaning that you do not have to check air pressure or fill them with air. Always check the tires for tears or other damage, however.

Troubleshooting

The wheelchair does not move although the lever is squeezed:

- Check whether the PowerStroll is switched on.
- Check whether the battery is fully charged.
- Check whether the connection between the battery and motor is correct.
- Check whether the brakes of the wheelchair are released.
- Check whether the drive wheel is in contact with the ground.
- **It is difficult to move the wheelchair by hand:**
 - Check whether the brakes of the wheelchair are released.
 - • Raise the PowerStroll.

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Drive Medical GmbH & Co. KG