TRAVELITE

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Table of contents

Preface	9
Declaration of conformity	9
Service life	9
Disposal	9
Identification plate	9
Reuse	9
Warranty	9
Parts of the product	10
Intended purpose	11
Indication	11
Scope of delivery	11
Safety instructions	11
Assembly	12
Folding Transport	12
Parking brake	13
Cleaning Maintenance	13
Technical Data	13

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.

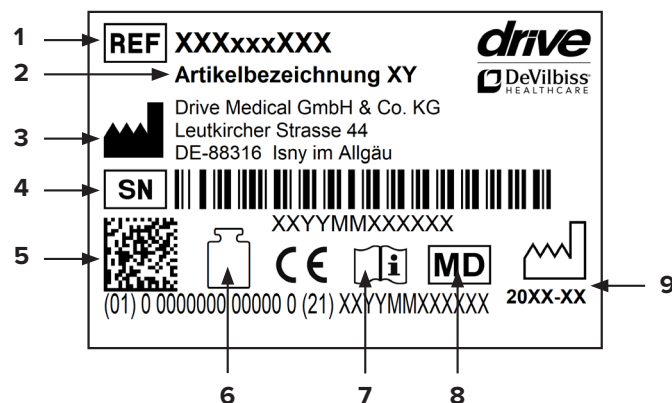
This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at www.drivedevilbiss.de

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others): wheels, brakes, handles and armrests

Parts of the product

Travelite

- | | | |
|--------------------|--------------------------------|--------------------------------|
| 1. Pushing handles | 5. Footrest | 9. Rear wheel |
| 2. Armrest | 6. Foot plate | 10. Parking brake |
| 3. Lifting strap | 7. Tilt aid | 11. Backrest locking mechanism |
| 4. Armrest release | 8. Front wheel with wheel fork | |

Intended purpose

Transport wheelchairs are intended for short-term use, indoors and outdoors (care facilities, hotels, airports, stations etc.) with an assistant/carer. They are used where there is significant to fully developed impairment of mobility/walking, with structural and/or functional damage of the lower extremities (including amputation, consequences of injury, musculoskeletal/neuromusculoskeletal movement disorders) and inability to propel a wheelchair independently (e.g. with limited strength and grip or lack of coordination of the arms/hands).

Indication

INDICATION:

Significant to full impairment of mobility/walking in the case of structural and/or functional damage to the lower extremities (including amputation, after-effects of injuries, musculoskeletal/neuromusculoskeletal movement disorders) and inability to independently use a wheelchair (e.g. in the case of limited strength and grip function or a lack of coordination ability of the arms/hands).

For short-term use as a transport wheelchair indoors and outdoors (nursing facilities, hotel, airport, train station, etc.) by an assistant/caregiver.

CONTRAINDICATION:

Push wheelchairs are suitable for outdoor use only to a limited degree. Due to the rear wheels which are smaller in comparison to push-rim wheelchairs, it is more difficult to get over obstacles such as curbs.

Scope of delivery

1 x	Transport wheelchair
1 x	Carrying bag with casters
1 x	Instructions for use

Safety instructions

To ensure proper and safe use, please be sure to observe the following information:

Use the transport wheelchair only on even and firm surfaces.

Stability and balance

Your transport wheelchair is designed such that it ensures the stability you need during normal daily activities. Each of your movements in the transport wheelchair has an effect on the center of gravity which in the worst case can lead to tipping of the transport wheelchair. To increase your safety, you must use the safety belt.

Reaching an object beyond the chair

The balance limits for reaching an object were determined through studies on a representative group of wheelchair users. Only the arms should be extended out beyond the wheelchair seat. The body and the head should remain within the limits of the seat.

Do not lean forward

Do not lean your chest over the armrest. To reach an object in front of you, you must lean forward and bend downwards. For this reason, you must use the steering wheels as an aid (by facing them forwards) in order to attain stability and balance. Accurate orientation of the wheels is important for your safety.

Do not lean back

Position the transport wheelchair as close as possible to the desired object so that you can easily reach it with outstretched arms while you remain seated in the chair in a normal position. Do not lean back too far over the backrest in any case since you can otherwise cause the chair to tip over.

Lateral transfer

This transfer can be performed alone, provided you have appropriate mobility and muscle tone.

Position the transport wheelchair as close as possible to the place where you wish to sit such that the steering wheels are pointing forwards. Secure the parking brakes of the rear wheels.

Use a transfer board, if possible.

Forward transfer

If you are able to stand up, you can transfer yourself forwards to another seat. If this movement is difficult for you, we recommend obtaining assistance from an attendant or caregiver to ensure a transfer that is as easy and safe as possible.

Secure both brakes and ensure that both steering wheels are pointing forwards. In doing so, be aware that you should not under any circumstances support yourself on the brakes during the transfer.

Swivel the footrests out of the way. Then grasp the top of the armrests, support yourself on your arms in order to easily get up and the attendant can assist you in completing the transfer.

Never stand on the footrests and never use the footrests as assistance when getting up.

Tilting, overcoming obstacles with an attendant

Getting up onto the sidewalk

The attendant positions the transport wheelchair facing forward in front of the sidewalk. He/she then tilts the transport wheelchair backwards towards him-/herself until the steering wheels reach up onto the sidewalk; he/she pushes the transport wheelchair forward until the

rear wheels are at the edge of the curb and pushes the transport wheelchair further until the rear wheels roll over the edge of the curb.

Getting down off the sidewalk

The attendant positions the transport wheelchair facing forward with the steering wheels at the edge of the curb and tilts it backwards towards him-/herself. Then he/she carefully pushes the transport wheelchair forwards until the rear wheels touch the street, then he/she gradually lowers the steering wheels to the ground.

Getting over stairs

The transport wheelchair should not be carried when in use by a patient.

To navigate stairs, the user must get out of the transport wheelchair and use the stairs, for example with the aid of an attendant. The transport wheelchair can then be carried up or down the stairs by the attendant.

- Do not attempt to lift the transport wheelchair by grasping any removable parts (for example, the armrests, leg supports or footrests).
- Avoid using an escalator.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.
- Protect the product from direct sunlight. Product parts can become hot and cause skin injuries if touched.
- Every serious incident which occurs in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.

Assembly

Unfold the transport chair by pushing the 2 sides of the sitting surface down. Ensure that the transport chair is completely unfolded.

Swivel the armrest inwards until it clicks into place. Ensure that the armrest has correctly clicked into place (**pic. 1**).

Pull the foot plate out of the footrest until it audibly clicks into place in the desired position (**pic. 2**).

Swivel the push handles up until they audibly click into place; only then are they securely locked (**pic. 3**).

Folding | Transport

Note: Fold the chair by following the unfolding directions in reverse order.

Unlock the push handle and fold it downwards (**pic. 4**).

Unlock the armrest and swivel it backwards (**pic. 5**).

Press both pins together and push the footrests together as far as possible.

Pull the lifting straps on the seat upwards. The chair will then fold automatically.

The product is not suitable for use as a seat in a motor vehicle. Users should always use the vehicle's seat!

Parking brake

The transport wheelchair comes with two parking brakes which may only be used when the chair is standing still.

To use the parking brake, press the brake lever firmly to the back until it noticeably clicks into place – the parking brake is now engaged. **(pic. 6).**

To release the parking brakes, pull the brake handles back up.

Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60°C for long periods of time (for example, use in a sauna, etc.).
- In the event of defects, contact your specialist dealer from whom you purchased this product.
- Check the condition of the tires regularly. They are “fail-safe”, meaning that you do not have to check air pressure or fill them with air. Always check the tires for tears or other damage, however.

Tecnical Data

Product	TRAVELITE
Article number	TC005
Total length	84 cm
Overall width	57 cm
Overall height	95 cm
Seat width x Seat depth	45 cm x 45 cm
Seat height	48 cm
Folding size width	29 cm
Weight	9,3 kg
Max. load	115 kg

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Drive Medical GmbH & Co. KG