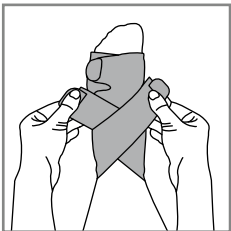
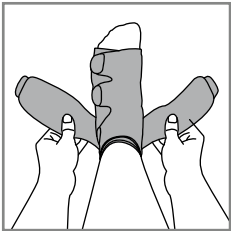
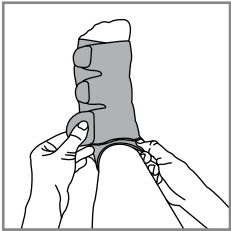
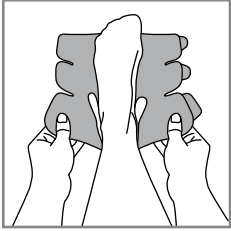
Juzo ACS Light Juzo Compression Wrap

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EN

Intended purpose

Adjustable Compression Devices ACD (Juzo ACS Light and Juzo Compression Wrap) are products used in compression therapy. Compression therapy accelerates venous return flow and reduces oedema. As a result, in addition to treating oedema of different origins, healing and prevention of venous ulceration is achieved. Juzo ACS Light and Juzo Compression Wrap are adjustable compression Wraps that are used as an alternative or in addition to conventional compression bandages. They are easy to put on and compression can be individually adjusted, thanks to the touch fasteners. They are characterised by high working pressure and can be worn throughout the day and night.

Notes for application

To ensure compression from distal to proximal, the different segments must be worn in combination. If this is not ensured, "window oedemas" may form. Your specialist retailer or therapist will be happy show you the correct way to use the product. You can also watch a film with detailed donning instructions at juzo.com

Juzo ACS Light

Put on your compression garment while sitting down to prevent injuries or falling. Before you put on the product, open all touch fasteners. For the foot, knee and hand wrap, the hook and loop Segments must be attached to the tabs before you wear it for the first time. Make sure that the label is always on the inside and the silicone border (if there is one) is at the top. There must not be any large folds during donning or wearing. If required,

you can increase the wrap's compression by adjusting the tabs from the bottom to the top. All Juzo ACS Light products can also be used without the incorporated lining.

Juzo ACS Light foot wrap (A)

1. In order to wear the Juzo ACS Light foot wrap in combination with the calf wrap, please put on the calf wrap first so that the foot wrap overlaps with it in the area of the ankle.
2. Attach the supplied touch fasteners halfway on the tab ends so that you can secure the tabs from the outside.
3. Position your foot so that your heel is in the designated opening.
4. Fasten the two small tabs around the forefoot, then the large tab around the ankle.
5. Now cross the supplied extender (reinforcement tab) diagonally over the instep or around the ankle, depending on where the oedema is located.
6. Check that the product exerts pleasant, clearly noticeable compression. If you feel pain or other problems, loosen the touch fasteners.

Juzo ACS Light calf wrap (B)

1. Juzo ACS Light calf wraps are available in left and right versions. The touch fasteners are integrated into the tab ends. The sewn-in adhesive border needs to be at the top.
2. Put your leg into the integrated lining. The calf wrap ends at ankle level.
3. Start by closing the inner row of tabs (3 or 4 tabs, depending on size) above the ankle. To do this, tighten the first tab around the leg and secure it with the touch fastener.
4. Continue closing the other tabs from the bottom to the top. The top tab should end approximately two centimetres below the back of the knee.
5. Proceed in the same way with the outer row of tabs (2 or 3 tabs, depending on size). Tip: if you find it difficult to put on the foot and calf wrap, we suggest placing your foot on a chair while standing. Alternatively, lie on a bed and raise your legs.
6. Check that the product exerts pleasant, clearly noticeable compression. If you feel pain or other problems, loosen the touch fasteners.

Juzo ACS Light knee wrap (C)

1. Attach the supplied touch fasteners halfway on the tab ends so that you can secure the tabs from the outside.
2. Put your leg into the integrated lining. To check the correct fit, make sure that the cut-out opening at the back of the knee wrap is at the back of the knee.
3. First, close the middle tabs to secure the wrap.
4. Then tighten the lower tabs around the leg and fix them with the touch fasteners, followed by the upper tabs.
5. Check that the product exerts pleasant, clearly noticeable compression. If you feel pain or other problems, loosen the touch fasteners.

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1. Juzo ACS Light thigh wraps are available in left and right versions. The touch fasteners are integrated into the tab ends. The sewn-in adhesive border needs to be at the top.
2. Put your leg into the integrated lining.

3. First, close the top tab to secure the wrap at groin level.
4. Continue closing the other tabs from the bottom to the top. Tighten them around the leg and fix them with the touch fasteners.
5. The supplied extender can be used as an addition for overheight at the front of the leg or the side.
6. Check that the product exerts pleasant, clearly noticeable compression. If you feel pain or other problems, loosen the touch fasteners.

Juzo ACS Light hand wrap (E)

1. If wearing the Juzo ACS Light hand wrap in combination with the arm wrap, please put on the arm wrap first so that the hand wrap overlaps with it in the area of the wrist.
2. Attach the supplied touch fasteners halfway on the tab ends so that you can secure the tabs from the outside.
3. Insert your thumb in the dedicated opening so that your palm initially remains free and the majority of the wrap covers the back of your hand.
4. First, tighten the small tab around the palm and secure it on the back of the hand using the hook and loop fastener.
5. Then tighten the wide tab around the wrist and close it outwards.
6. The supplied foam pads can be placed on the back of the hand or on the palm, if required.
7. Check that the product exerts pleasant, clearly noticeable compression. If you feel pain or other problems, loosen the touch fasteners.

Juzo ACS Light arm wrap (F)

1. Juzo ACS Light arm wraps are available in left and right versions. The touch fasteners are integrated into the tab ends.
2. Put your arm into the integrated lining with the palm facing upwards. Now position the upper edge of the wrap below your armpit. The comfort zone of the arm wrap is located in the crook of the arm.
3. Bend your arm at a 90 degree angle and start by closing the tabs from the wrist. Tighten them towards the outside of the arm.
4. Continue closing the other tabs from the bottom to the top.
5. The supplied foam pad can be inserted into the dedicated pocket in the elbow area if required.
6. Check that the product exerts pleasant, clearly noticeable compression. If you feel pain or other problems, loosen the touch fasteners.

Juzo ACS Light additional products

The following additional products are available:

- Juzo ACS Light hip band (J711)
- Juzo ACS Light midline band, upper extremity (J712)
- Juzo ACS Light touch fasteners (J713)
- Juzo ACS Light pads (J714)
- Juzo ACS Light anti-slip band (J715)
- Juzo ACS Light extender (J716)

Your specialist retailer or therapist will be happy show you the correct way to use the product.

Juzo Compression Wrap (G)

Juzo Compression Wraps can be worn either with the beige or the black side facing outwards (please note the information about the hand portion). Put on your compression garment while sitting down to prevent injuries or falling. Before putting on the wrap, secure the touch fasteners supplied on the insides of the tabs. First, fold all tabs inwards to

prevent the touch fasteners attaching accidentally. There must not be any large folds during donning or wearing. If required, you can increase the wrap's compression by adjusting the tabs from the bottom to the top.

Juzo Compression Wrap – foot wrap (H)

1. In order to wear the Juzo Compression Wrap foot wrap in combination with the calf wrap, please put on the calf wrap first so that the foot wrap overlaps with it in the area of the ankle.
2. Position your foot so that your heel is in the designated opening.
3. First, tightly close the front tab around the forefoot. Continue with the other tabs from the bottom to the top.
4. The individual tabs as well as the combined wraps must overlap without any gaps.
5. Check that the product exerts pleasant, clearly noticeable compression. If you feel pain or other problems, loosen the touch fasteners.

Juzo Compression Wrap – calf wrap (I)

1. Insert your leg into the supplied liner stocking so that your leg is covered up to just below the knee.
2. Put your lower leg into the integrated lining of the Juzo Compression Wrap.
3. Position the Juzo Compression Wrap so that its upper edge lies two finger widths below your knee. The seam should run along the middle of the calf.
4. First, close the top tab to secure the segment.
5. Now close the two middle tabs at the same time, then the bottom tab.
6. Then alter the wrap's compression by adjusting the tabs from the bottom to the top. The individual tabs as well as the combined segments must overlap without any gaps.
7. Check that the product exerts pleasant, clearly noticeable compression. If you feel pain or other problems, loosen the touch fasteners. Tip: if you find it difficult to put on the product, we suggest you place your foot on a chair while standing. Alternatively, put your leg up while sitting.

Juzo Compression Wrap – knee wrap (J)

1. Position the Juzo Compression Wrap so that the horizontal seam is in the centre of the back of your knee.
2. First, close the top tab to secure the segment.
3. Now close the bottom tab.
4. Then alter the wrap's compression by adjusting the tabs from the bottom to the top. The individual tabs as well as the combined segments must overlap without any gaps.
5. Check that the product exerts pleasant, clearly noticeable compression. If you feel pain or other problems, loosen the touch fasteners.

Juzo Compression Wrap – thigh wrap (K)

1. Position the Juzo Compression Wrap so that the bottom end extends two finger widths over the knee and the horizontal seam is in the centre of the back of your knee.
2. First, close the top tab to secure the segment.
3. Now close the bottom tab.
4. Then alter the wrap's compression by adjusting the tabs from the bottom to the top. The individual tabs as well as the combined segments must overlap without any gaps.

5. Check that the product exerts pleasant, clearly noticeable compression. If you feel pain or other problems, loosen the touch fasteners.

Juzo Compression Wrap – arm wrap (L)

1. Put your arm into the integrated lining of the Juzo Compression Wrap until the cross seam runs across your elbow.
2. First, close the bottom tab on your wrist.
3. Continue with the other tabs from the bottom to the top.
4. Then alter the wrap's compression by adjusting the tabs from the bottom to the top. The individual tabs as well as the combined segments must overlap without any gaps.
5. Check that the product exerts pleasant, clearly noticeable compression. If you feel pain or other problems, loosen the touch fasteners.

Juzo Compression Wrap – hand wrap (M)

1. If wearing the Juzo Compression Wrap hand wrap in combination with the arm wrap, please put on the arm wrap first so that the hand wrap overlaps with it in the area of the wrist.
2. The Juzo Compression Hand Wrap is designed to be reversible. The hand wrap product designation (UHR for right or UHL for left) applies to wearing the product with the beige side facing outwards.
3. To don the hand wrap, put your thumb in the thumb stub so that the short side of the wrap is on the palm side. Wrap the garment around the hand and bring the thumb through the thumb hole.
4. Attach the fasteners, beginning with the wrist strap. Then close the tab on the back of your hand.
5. Then alter the wrap's compression by adjusting the tabs from the bottom to the top. Check that the product exerts pleasant, clearly noticeable compression. If you feel pain or other problems, loosen the touch fasteners.

Juzo Compression Wrap additional products

The following additional products are available:

- Touch fasteners – foot, hand (6000F1)
- Touch fasteners – calf (6000F2)
- Touch fasteners – calf long, arm (6000F3)
- Touch fasteners – knee, thigh, arm long (6000F4)
- Touch fasteners – thigh long (6000F5)
- Touch fasteners – foot long (6000F6)
- Extender 120 cm x 8,5 cm (6000EXT)

Your specialist retailer or therapist will be happy show you the correct way to use the product.

Please note:

Only wear Adjustable Compression Devices ACD if they have been prescribed by a doctor. These systems should only be dispensed by properly trained staff at a medical retailer. In addition to use by medical specialists, ACD are also suited for long-term and self-therapy if patients or their carers 1) are able to do so based on their cognitive and motor skills and 2) have been instructed in the correct use by medical specialists. ACD have been exclusively designed for the specific body part (foot, lower leg, knee, thigh, hand and arm) in accordance with the relevant product designations. Based on their short-stretch properties, ACD have mechanically limited elasticity. The desired treatment effect is based on correct use. After adjusting the hook and loop fasteners, the user must therefore ensure each time that

the product feels pleasant, noticeable compression. If you feel pain or notice any other problems, loosen the touch fasteners immediately. Progress in treatment success may require a remeasurement of body dimensions and, if necessary, a new product. Please contact your specialist retailer in the event of damage to your medical adaptive compression system or its accessories. The sewn-in textile label (if present) is important for the identification and traceability of the product. We therefore ask you not to remove the label under any circumstances.

Washing and care instructions

Please note the care instructions on the textile label sewn into your compression garment. Prior to washing, either remove or close all touch fasteners. The products can be washed daily in the machine at 30 °C using a laundry detergent for delicate fabrics. Tip: A laundry net additionally protects the product. Please do not use fabric softener. Never use bleach. Juzo ACS Light products can be dried in a tumble-dryer. Do not tumble dry Juzo Compression Wrap products. You can reduce drying time by placing the products on a thick towel after washing, tightly rolling the towel up and firmly squeezing out excess moisture. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Juzo ACS Light and Juzo Compression Wrap products must not be ironed. The products are not suitable for dry cleaning. The intervals between washing can be extended by wearing a liner stocking (e.g. Juzo Liner 6082 AD, Juzo Liner 6082 AG).

Treatment and disinfection of Juzo ACS Light

This product has been successfully tested in accordance with DIN ISO TS 15883-5, SOD 19-002, EN ISO 17664, SOD 19-01 and the Robert Koch Institute "Hygiene Requirements for the Reprocessing of Medical Devices" guideline. The Juzo ACS Light can be quantifiably treated, cleaned and disinfected by observing the instructions in the guideline of the German Society for Hospital Hygiene (DGKH) and the German Society for Sterile Supply (DGSV).

Material composition

For precise details, please see the textile label sewn into your compression garment.

Storage information and shelf life

Store in a dry place and protect from sunlight. Standard products have a maximum shelf life of 36 months. The compression garment then has a maximum usage period of 6 months. This period depends on correct handling (e.g. during caring, donning and doffing) and normal use of the products. The usage period of the medical device is printed on the packaging with an hourglass symbol.

Juzo ACS Light and Juzo Compression Wrap indications

Lymphoedema, Lipoedema, Venous oedema, Venous leg ulcers, Post-traumatic oedema, Thrombosis prophylaxis in mobile patients, Post-operative oedema, Deep vein thrombosis, Post-thrombotic syndrome, Chronic venous insufficiency, Varicosis, Congestion due to immobility, Work-related oedema (work standing up / sitting down), Before and after vein surgery, Dermatoliposcleriosi

Juzo ACS Light and Juzo Compression Wrap – Contraindications

Advanced peripheral arterial occlusive disease (if one of these parameters applies ABPI < 0.5, ankle artery pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO₂ < 20 mmHg instep). When inelastic

materials are used, compression garments can still be worn with an ankle artery pressure between 50 and 60 mmHg with continual clinical supervision. Decompensated cardiac insufficiency (NYHA III + IV), Septic phlebitis, Phlegmasia coerulea dolens.

In the following cases, the decision about treatment should be made considering the benefits and risks as well as choosing the most suitable compression option:

Pronounced weeping dermatoses, Incompatibility with pressure or product ingredients, Severe sensitivity disorders in the extremities, Advanced peripheral neuropathy (e.g. in cases of diabetes mellitus), Primary chronic polyarthritits

Important

Open wounds in the area to be treated must be covered with a sterile wound dressing adapted to exudate management before the compression garment can be applied.

Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

Liability

In case of improper use (including independent repairs or third-party providers), the manufacturer cannot guarantee the product's conformity. The claim to warranty and substitute performance is voided.

Side effects

There are no known side effects when using this product correctly. However, if you notice any adverse changes (e.g. skin irritation) during the prescribed use, please consult your doctor, therapist or medical retailer immediately. If you know you are allergic to any product ingredients, please consult your doctor or therapist before using it. If your symptoms worsen during the time you are wearing the product, please take off the product and consult a medical specialist immediately. The manufacturer does not accept any liability for damage / injuries resulting from improper use or misuse. In the event of complaints related to this product, such as damage to the material or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017 / 745 (MDR).

Disposal

As a rule, the product is to be destroyed via the conventional disposal route after the end of use. Please observe the applicable local/national regulations.

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**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany · info@juzo.com · juzo.com
**Berro AG** · Austrasse 7 · 9055 Bühler · Suisse
UK responsible person: Juzo UK Ltd