## Juzo SleepCompress

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EN

### **Intended purpose**

The Juzo SleepCompress effectively supports compression therapy, both at night and during daytime rest phases. Compression therapy reduces oedema and accelerates venous return flow. Vertical foam channels form a unique surface structure that supports lymphatic drainage.

## Notes for use

Your specialist retailer or your therapist will be happy to show you how to use the product correctly. The hook and loop segments must be attached to the tabs before you wear the product for the first time. Make sure the label is on the inside. There must not be any large folds during donning or wearing. If required, you can increase the wrap's compression by adjusting the tabs from the bottom to the top. The Juzo SleepCompress can also be used without the incorporated lining.

## Arm (A)

1. Juzo SleepCompress arm are available in left and right versions. The touch fasteners must be attached to the product.
2. Put your arm into the integrated lining with your palm facing upwards. Now position the upper edge of the wrap below your armpit. The comfort zone of the arm wrap is located in the crook of the arm.
3. Bend your arm at a 90 degree angle and start by closing the tabs from the hand. Tighten them towards the outside of the hand. Continue closing the other tabs from the bottom to the top.
4. Check that the product exerts pleasant, noticeable compression. If you feel pain or other problems, loosen the touch fasteners.

## Calf (B)

1. Juzo SleepCompress below-knee is available in left and right versions. The touch fasteners must be attached to the product.
2. Put your leg into the integrated foot lining. Position your foot so that your heel is in the designated comfort area.
3. Start by closing the tabs on the foot. Tighten them towards the outside of the leg. Continue closing the other tabs from the bottom to the top.
4. Check that the product exerts pleasant, noticeable compression. If you feel pain or other problems, loosen the touch fasteners.

## Leg (C)

1. Juzo SleepCompress Leg is available in left and right versions. The touch fasteners must be attached to the product.
2. Put your leg into the integrated foot lining. Position your foot so that your heel is in the designated comfort area.
3. Start by closing the tabs on the foot. Tighten them towards the outside of the leg. Continue closing the other tabs from the bottom to the top.

4. Check that the product exerts pleasant, noticeable compression. If you feel pain or other problems, loosen the touch fasteners.

## Please note:

Only wear Juzo SleepCompress if it has been prescribed by a doctor. The products should only be dispensed by properly trained staff at a medical retailer. Further treatment success or a change in condition will require re-measurement and potentially a new product. Please contact your specialist retailer in the event of damage. Do not repair the product yourself – this may affect its quality and medical effectiveness. The sewn-in textile label (if present) is important for the identification and traceability of the product. We therefore ask you not to remove the label under any circumstances.

## Washing and care instructions

Wash the Juzo SleepCompress after each use. Please note the care instructions on the textile label sewn into your product. Prior to washing, close all touch fasteners or, if this is not possible, remove them. The products can be machine-washed at 60°C using a laundry detergent for delicate fabrics. Tip: a mesh laundry bag additionally protects the product. Please do not use fabric softener. Never use bleach. Before drying, stretch the items while they are wet. You can reduce drying time by placing the Juzo SleepCompress on a thick towel, rolling the towel up tightly and firmly pressing out excess moisture. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Juzo SleepCompress can be tumble-dried but must not be ironed.

## Treatment and disinfection

(for hospital use)

This product has been successfully tested in accordance with DIN ISO TS 15883-5, SOD 19-002, EN ISO 17664, SOD 19-01 and the Robert Koch Institute "Hygiene Requirements for the Reprocessing of Medical Devices" guideline. The Juzo SleepCompress can be quantifiably treated, cleaned and disinfected in compliance with the provisions of the guideline compiled by the German Society for Hospital Hygiene (DGKH) and the German Society for Sterile Supply (DGSV).

## Material composition

For precise details, please see the textile label sewn into your compression garment.

## Storage information and shelf life

Store in a dry place and protect from sunlight. Standard products have a maximum shelf life of 36 months. The

compression garment then has a maximum usage period of 6 months. This period depends on correct handling (e.g. during caring, donning and doffing) and normal use of the products. The usage period of the medical device is printed on the packaging with an hourglass symbol.

## Indications

Lymphoedema, venous oedema, lipoedema, post-traumatic oedema, post-operative oedema

## Contraindications

Advanced peripheral arterial disease (if one of these parameters applies: ABPI < 0.5, ankle artery pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO<sub>2</sub> < 20 mmHg instep). Decompensated heart failure (NYHA III + IV), septic phlebitis, phlegmasia cerulea dolens.

In the following cases, the decision about treatment should be made considering the benefits and risks as well as choosing the most suitable compression option:

Pronounced weeping dermatoses, intolerance to the compression material, severe sensory disorders of the extremities, advanced peripheral neuropathy (e.g. diabetes mellitus), primary chronic polyarthritis.

## Important

Open wounds in the area to be treated must be covered with a sterile wound dressing adapted to exudate management before Juzo SleepCompress can be applied.

Softcompress cannot accept liability for any failure to comply with these contraindications.

## Side effects

There are no known side effects when using this product correctly. However, if you notice any adverse changes (e.g. skin irritation) during the prescribed use, please consult your doctor, therapist or medical retailer immediately. If you know you are allergic to any product ingredients, please consult your doctor or therapist before using it. If your symptoms worsen during the time you are wearing the product, please take off the product and consult a medical specialist immediately. The manufacturer does not accept any liability for damage / injuries resulting from improper use or misuse.

In the event of complaints related to this product, such as damage to the material or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017 / 745 (MDR).

## Disposal

You can dispose of your Juzo SleepCompress in the usual way. There are no specific disposal instructions.

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**Berro AG** · Austrasse 7 · 9055 Bühler · Suisse
**UK responsible person: Juzo UK Ltd**