
**X** Choice

 **Juzo**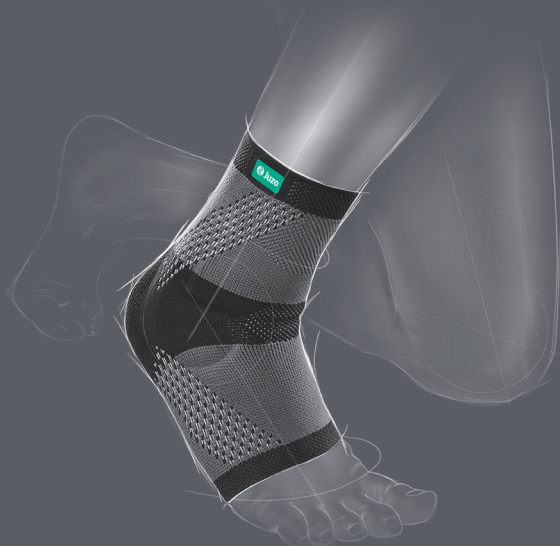

JuzoFlex

## Malleo Xtra

Sprunggelenkbandage · Ankle support

---

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użycia · Инструкция по эксплуатации · Návod na používanie · Navodila za uporabo · تعليمات الاستخدام

---

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EN

## Intended purpose

The JuzoFlex Malleo Xtra provides anatomical guidance for the affected ankle, thus relieving it during movement. Medical compressive active supports have a proprioceptive effect. They improve self-perception and can compensate for muscle imbalances. They alleviate pain and support the removal of fluids from the tissue, thanks to their rheological effect.

## How to put on the product correctly:

Put on your compression support while you're sitting down to prevent injuries or falling.

1. Grasp the support on the upper edge and pull it over your ankle.
2. The correct fit of your support is guaranteed when it firmly surrounds the ankle without applying too much (painful) pressure on the ankles (malleoli). Fine adjustment of the support may be made at any time.

Caution: rough, unfiled finger nails or sharpedged rings etc. may damage the knitted fabric. Toe nails and calluses can cause damages, as can the defective inner lining of shoes.

## Please note:

Only wear your support when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The support can only be fully effective when it has been put on correctly and is being worn in accordance with treatment recommendations. The support is largely resistant to oils, ointments, skin moisture and other environmental influences. The durability of the material may be affected

by certain combinations of the above. We therefore recommend that it be examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the support yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the support is not suitable for treating more than one patient. The sewn-in textile label is important for the identification and traceability of the product. We therefore ask you not to remove the label under any circumstances.

## Washing and care instructions

Please note the care instructions on the textile label sewn into your stockings. Wash the supports separately the first time (colour may run). Your support should be washed every day or between uses on a gentle or delicate wash cycle (30 °C). Tip: a laundry net provides additional protection for the knitted fabric. Please do not use fabric softener! When hand washing, rinse well and do not wring. We recommend the gentle Juzo special detergent. You can reduce drying time by placing the support on a thick towel after washing, tightly rolling the towel up and firmly pressing out excess moisture. Then hang the support up loosely. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Juzo supports are not suitable for dry cleaning. The support is not suitable for tumble drying.

## Material composition

For precise details, please see the textile label that is sewn into your product.

## Storage information and shelf life

The medical retailer or doctor prescribing the support will recommend exactly how long it should be used for. If the product is being worn and washed every day, its maximum usage duration will be 6 months. If the product is used on an irregular basis, the wearing duration will be longer, up to a maximum of 12 months. Based on individual use, product wear may differ. We therefore recommend that the product is checked by a medical specialist in the event of suspected changes. Store the product in a dry place and protect it from sunlight. Supports have a maximum shelf life of 42 months. The usage duration of the medical product is printed on the box with an hour-glass symbol. Body measurements may change based on the patient's condition and individual circumstances. We therefore recommend that body measurements are checked regularly by a medical specialist.

## Indications

Accelerated reabsorption of oedemas, effusions and haematomas, postoperative recovery, degenerative joint disorders (ankle osteoarthritis), irritation of the ankle joint, early mobilisation, mobilisation phase after lesions of the ankle ligaments.

## Contraindications

If you have any of the following conditions, you should only wear the support after consulting with your doctor: Lymph drainage disorders and non-specific soft tissue

swelling outside the area of application, sensory disturbances and impaired circulation in parts of the body to be treated, skin conditions, skin conditions, skin irritation and injuries in the area to be treated; open wounds in the treatment area must be covered with sterile material. Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

### Side effects

There are no known side effects when using this product correctly. However, if you notice any adverse changes (e.g. irritation of the skin) during the prescribed use, please consult your doctor or medical retailer immediately. If you know that you are allergic to one or more of the product ingredients, please consult your doctor before using the product. If your symptoms worsen during the time you are wearing the product, please take off the product and consult a medical specialist immediately. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse. In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

### Disposal

As a rule, the product is to be destroyed via the conventional disposal route after the end of use. Please observe the applicable local/national regulations.

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**Berro AG** · Austrasse 7 · 9055 Bühler · Suisse
**UK responsible person: Juzo UK Ltd**