JuzoPro

Patella Xtec Plus

Knie-Softorthese · Knee soft orthosis

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użycia · Инструкция по эксплуатации · Návod na používanie · Navodila za uporabo · تعليمات الاستخدام

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EN

Intended purpose

The compressive textile JuzoPro Patella Xtec Plus knee orthosis gives the patella gentle guidance thanks to pads. Additional adjustable tensioning elements are intended to exert a positive influence on the path of the patella.

How to put on the product correctly:

Put on your orthosis while you're sitting down to prevent injuries or falling.

1. Pull the knee orthosis over your forefoot up to the right place on your knee.
2. Then position the knee orthosis so that the patella (kneecap) is enclosed by the patella pad.
3. You can strengthen the effect on the patella by individually adjusting the strap fastening. Please do this when bending the knee slightly.

The correct fit of the orthosis is assured when it encloses the ankle tightly but does so without exerting a too strong (painful) pressure. The orthosis can be readjusted at any time by repeating steps 2 and 3 at any time.

Caution: rough, unfiled finger nails or sharpedged rings etc. may damage the knitted fabric. Toe nails and calluses can also cause damages.

Please note:

Only wear your orthosis when prescribed by your physician. Medical aids should only be dispensed by suitably trained staff of a medical products supplier. The orthosis can only be fully effective when it has been put on correctly and is being worn in accordance with treatment recommendations. The orthosis is largely resistant to oils, ointments, skin moisture and other environmental influences. The durability of the material may be affected by certain combinations of the above. We therefore recommend that it gets examined regularly by the medical products supplier. Should the product become damaged, please consult your specialist supplier. Do not repair the orthosis yourself – this could compromise its quality and medical efficacy. For hygienic reasons, the orthosis is not suitable for treating more than one patient. The sewn-in textile label is important for the identification and traceability of the product. We therefore ask you not to remove the label under any circumstances.

Washing and care instructions

Please note the care instructions on the textile label attached to your orthosis. Wash the orthosis daily or after each use on a gentle or delicates cycle at approx. 30 °C. Wash the orthosis separately the first time (colours may run). When hand washing, please rinse the orthosis well and leave it to dry – lying flat if possible – at room temperature. Do not wring it out. We recommend using the gentle Juzo Special Detergent for this. You can reduce the drying time by placing the orthosis on a thick towel, rolling the towel up tightly and firmly pressing out excess moisture. Do not leave it in the towel and do not dry it on a radiator or in the sun. The orthosis is not suitable for tumble drying. This Juzo product is not suitable for dry cleaning.

Tip: a laundry net provides additional protection for the knitted fabric. Please do not use fabric softener!

Material composition

For specific details please refer to the textile label sewn onto your orthosis.

Storage information and usage duration

The medical retailer or doctor prescribing the orthosis will recommend exactly how long it should be used for. If the product is being worn and washed every day, its maximum usage duration will be 6 months. If the product is used on an irregular basis, the wearing duration will be longer, up to a maximum of 12 months. Based on individual use, product wear may differ. We therefore recommend that the product is checked by a medical specialist in the event of suspected changes. Store the product in a dry place and protect it from sunlight. Orthosis have a maximum shelf life of 42 months. The usage duration of the medical product is printed on the box with an hour-glass symbol. Body measurements may change based

on the patient's condition and individual circumstances. We therefore recommend that body measurements are checked regularly by a medical specialist.

Indications

Patellofemoral pain syndrome in cases of muscular imbalance, patellar maltracking, patellar lateralisation, hypermobile patella, misalignment, patella tip syndrome (jumper's knee), Osgood-Schlatter disease, following patellar tendon injuries, second phase after patellar stabilisation surgery, patella alta (high-riding patella).

Contraindications

Only in consultation with your doctor, the orthosis should be worn for the following conditions: Arterial circulation problems, all forms of varicose veins, post-thrombotic conditions, ulcerations, recently suffered or recurring thrombophlebitis, permanent lower leg oedemas, skin conditions or skin irritation; open wounds in the treatment area must be covered with sterile material. In addition, a compression stocking up to the knee (AD) must be worn by patients suffering from varicose veins or oedema.

Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

Side effects

There are no known side effects when using this product correctly. However, if you notice any adverse changes (e.g. irritation of the skin) during the prescribed use, please consult your doctor or medical retailer immediately. If you know that you are allergic to one or more of the product ingredients, please consult your doctor before using the product. If your symptoms worsen during the time you are wearing the product, please take off the product and consult a medical specialist immediately. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse.

In the case of complaints related to this product, such as damage to the knitted fabric or problems with the fit, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration of the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

As a rule, the product is to be destroyed via the conventional disposal route after the end of use. Please observe the applicable local/national regulations.

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**Julius Zorn GmbH** · Juliusplatz 1 · 86551 Aichach · Germany · info@juzo.com · juzo.com
**Berro AG** · Austrasse 7 · 9055 Bühler · Suisse
UK responsible person: Juzo UK Ltd