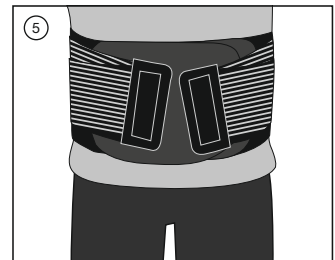
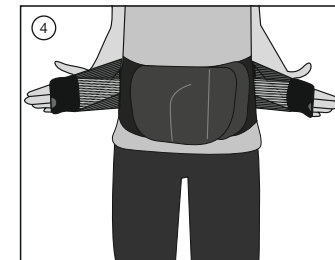
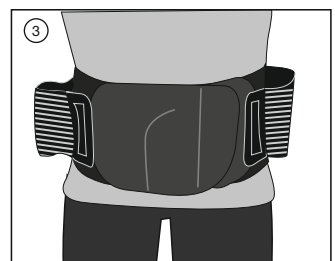
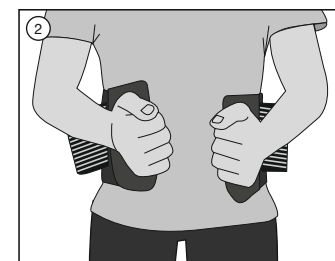
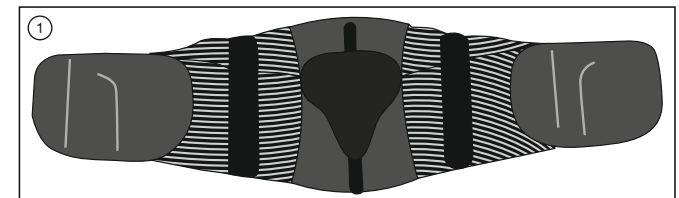
Cellacare® Lumbal Classic

Rückenorthese
Orthèse dorsale
Back orthosis
Órtesis dorsal
Rugorthese
Ortëza páteře

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en Instructions for Use

Dear customer,

Thank you for choosing an L&R
product.

Please read the instructions for use
carefully. If you have any questions,
please contact your physician or
specialist shop.

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Marquage CE initial : 2014

depending on the patient's medical
condition.

Usage

1. Apply the orthosis around the waist (Fig.
2). Close the orthosis at the front by
using the hook-and-loop fastener (Fig. 3).
2. Check the fit of orthosis and removable
back insert. If required, increase or
reduce the pressure by pulling harder or
less hard when closing the orthosis.
3. Pull the elastic additional side straps
forward simultaneously using the hand
grips (Fig. 4). Attach onto the large hook-and-loop
area, so that there is a pleasant feeling of
support in the back area.
4. Check the fit of the orthosis. If required,
increase or reduce the pressure by pulling
harder or less hard when closing the
additional side straps (Fig. 5).

Care recommendations

Remove the insert. Clean the insert
separately with a wet towel. Machine
wash the orthosis at 30 °C (delicate cycle)
with like colours. Do not bleach. Allow
Cellacare® Lumbal Classic to air-dry away
from heat sources; do not dry in a dryer.
Do not iron and do not dry clean. Fasten
the hook-and-loop closures and if possible
use a mesh bag. Clean the hook-and-loop
closures regularly so that they retain their
ability to fasten tightly, and to preserve the
performance of the product.

Storage and transport conditions

Cellacare® Lumbal Classic must be kept
away from light and moisture (preferably in
the original packaging).

Disposal

In Europe, product waste can be assigned
a waste code from Chapter 18 01 and
packaging waste a waste code from
Chapter 15 01 of the Directive on the
European Waste Catalogue (Waste
Catalogue Ordinance – AVV). Packaging
labelled as recyclable should be taken to
the applicable national recycling systems.

Side effects:

In very rare cases, skin irritations and / or
allergies may occur.

Warnings and precautions:

- Do not use Cellacare® Lumbal Classic
unless directed by a health care
professional. The first time patients use
the product they need to have a health
care professional show them how to put
it on correctly.
- The health care professional will
determine how long the product is to be
worn per day, based on the indications
and healing progress.
- The health care professional will
determine if the insert is needed,

- Cellacare® Lumbal Classic shall only be
used on intact skin. Wounds have to be
covered with a suitable wound dressing
before use.
- In case of skin diseases and / or injuries
in the application area (open, non-intact
or injured skin) please contact your
treating physician / seek medical advice
before application of the product.
- In case of pregnancy, the patient's
treating physician needs to give the
approval before using the product.
- Apply the product in accordance with
the instructions for use and illustrations
provided. Please check the positioning
of the product during use and correct as
necessary.
- In case of discomfort or problems,
consult a health care professional.
- Cellacare® Lumbal Classic must be
removed during longer periods of rest
(e.g. when sleeping) to avoid undesirable
congestive effects.
- If local swelling or numbness occurs
in the part of the body being treated,
the product must be removed and a
medically qualified health care
professional consulted.
- Cellacare® Lumbal Classic is intended for
use on a single patient only.
- The product's durability may be
compromised by certain factors, e.g.
objects with sharp edges or damage to
the hook-and-loop fastener. To prevent
this, the hook-and-loop fastener should
always be fastened, for example when
the product is not being worn or when
being washed.
- Cellacare® Lumbal Classic must not be
used beyond the last day of the month
indicated on the packaging (expiry date).
- If there are excessive signs of wear
that could influence the performance
of the product, consult a healthcare
professional and replace the product if
necessary.

Usage information:

Preparation

Preparation

1. Apply Cellacare® Lumbal Classic while
standing.
2. For hygienic reasons wearing a
vest underneath the orthosis is
recommended.
3. Open the front hook-and-loop fastener
and flatten out the orthosis in front
of you. If the insert is separated from
the orthosis, attach the insert to the
designated loop strip on the inside of the
orthosis (Fig. 1).

Initial CE mark: 2014

4. Check that the elastic straps are attached
without tension to the dedicated hand
grip area (Fig. 3).

Usage

1. Apply the orthosis around the waist (Fig.
2). Close the orthosis at the front by
using the hook-and-loop fastener (Fig. 3).
2. Check the fit of orthosis and removable
back insert. If required, increase or
reduce the pressure by pulling harder or
less hard when closing the orthosis.
3. Pull the elastic additional side straps
forward simultaneously using the hand
grips (Fig. 4). Attach onto the large hook-and-loop
area, so that there is a pleasant feeling of
support in the back area.
4. Check the fit of the orthosis. If required,
increase or reduce the pressure by pulling
harder or less hard when closing the
additional side straps (Fig. 5).

Care recommendations

Remove the insert. Clean the insert
separately with a wet towel. Machine
wash the orthosis at 30 °C (delicate cycle)
with like colours. Do not bleach. Allow
Cellacare® Lumbal Classic to air-dry away
from heat sources; do not dry in a dryer.
Do not iron and do not dry clean. Fasten
the hook-and-loop closures and if possible
use a mesh bag. Clean the hook-and-loop
closures regularly so that they retain their
ability to fasten tightly, and to preserve the
performance of the product.

Storage and transport conditions

Cellacare® Lumbal Classic must be kept
away from light and moisture (preferably in
the original packaging).

Disposal

In Europe, product waste can be assigned
a waste code from Chapter 18 01 and
packaging waste a waste code from
Chapter 15 01 of the Directive on the
European Waste Catalogue (Waste
Catalogue Ordinance – AVV). Packaging
labelled as recyclable should be taken to
the applicable national recycling systems.

Side effects:

In very rare cases, skin irritations and / or
allergies may occur.

Warnings and precautions:

- Do not use Cellacare® Lumbal Classic
unless directed by a health care
professional. The first time patients use
the product they need to have a health
care professional show them how to put
it on correctly.
- The health care professional will
determine how long the product is to be
worn per day, based on the indications
and healing progress.
- The health care professional will
determine if the insert is needed,

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