Cellacare® DigiFinger Classic

Orthese zur Immobilisierung des Handgelenks, der Hand, der Finger
Orthèse pour l'immobilisation du poignet, de la main, des doigts
Orthesis for immobilisation wrist - hand - fingers
Órtesis para la inmovilización de la muñeca, la mano y los dedos
Ortesi per l'immobilizzazione del polso, della mano, delle dita
Orthese voor het immobiliseren van de pols, hand, vingers

1 Einzelner Patient – mehrfach anwendbar · un seul patient – à usage multiple · single patient – multiple use · un solo paciente – uso múltiple · singolo paziente – uso multiplo · één patiënt – meervoudig gebruik

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en Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics

Cellacare® DigiFinger Classic is an orthosis for correction and immobilisation of the finger joints, to provide stability to the injured finger and to restrict its movements.

Product composition

Polyamide, polyester, polyurethane, aluminium, cotton

Intended purpose

Orthosis for orthopaedic correction and immobilisation of the finger joints

Indications

Sprained fingers
Dislocation of the fingers
Pre- or post-operative immobilisation
Immobilisation of the 2nd, 3rd and 4th, 5th metacarpophalangeal joints

Contra-indications

• Known allergy and / or hypersensitivity to any of the product components

Side effects

In very rare cases, skin irritations and / or allergies may occur.

Warnings and precautions

- Do not use Cellacare® DigiFinger Classic unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.
- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- Cellacare® DigiFinger Classic shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before application of the product.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- If local swelling or numbness occurs in the part of the body being treated, Cellacare® DigiFinger Classic must be removed and a medically qualified health care professional consulted.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.
- Cellacare® DigiFinger Classic is intended for use on a single patient only.
- The device's durability may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook-and-loop fasteners. To prevent this, the hook-and-loop fasteners should always be fastened, for example, when the product is not being worn or when being washed.
- Cellacare® DigiFinger Classic must not be used beyond the last day of the month indicated on the packaging (expiry date).
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Usage information

Preparation

1 - Open all the hook-and-loop fasteners and lay the product flat (Fig 1). There are 4 straps, 2 of which fixed (a and b) and 2 removable (c and d), to optimise support and adjustment.

Usage

1 - Slide the orthosis on the affected finger in association with the neighbouring (support) finger (Fig 2).

Make sure that the fingers are down to the end of the orthosis and that the syndactyly strap (c) is positioned at the level of the joint of the phalanges.

2 - Tighten the strap (a) around the wrist (Fig 3).

3 - Tighten the distal strap (b) at the tip of the fingers (Fig 4).

4 - Tighten the syndactyly strap (c) (Fig 5)

5 - Tighten the hand strap (d) (Fig 6)

Readjust the fit of the straps and the aluminium stay if necessary (Fig 7).

6 - In addition, the syndactyly strap could be used alone for resuming activities (Fig 8).

Care recommendations

Machine wash the orthosis at 30 °C (delicate cycle) with like colours. Do not bleach. Allow Cellacare® DigiFinger Classic to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean.

Close the hook-and-loop fasteners and if possible use a mesh bag. Clean the hook-and-loop closures regularly so that they retain their ability to fasten tightly, and to preserve the performance of the product.

Storage and transport conditions

Cellacare® DigiFinger Classic must be kept away from light and moisture (preferably in the original packaging).

Disposal

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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