
# Cellacare® Materna Comfort

Rückenorthese für die Schwangerschaft  
Orthèse dorsale pour la femme enceinte  
Maternity back orthosis  
Órtesis de espalda para el embarazo  
Ortesi dorsale per la gravidanza  
Rugorthese voor tijdens de zwangerschap  
Zádová ortéza pro těhotné

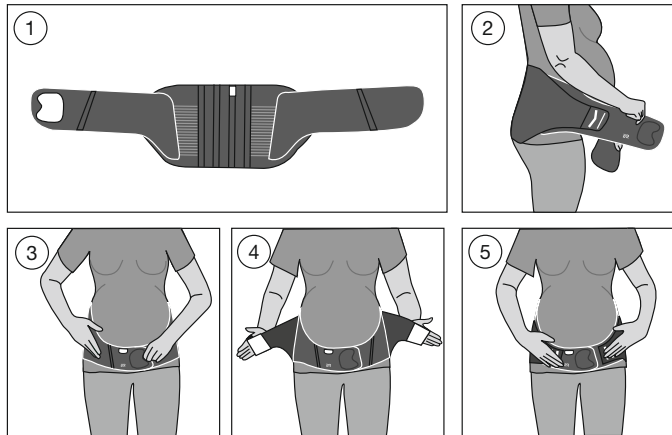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## en Instructions for Use

**Dear customer,**

**Thank you for choosing an L&R product.**

**Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.**

**Product description and performance characteristics:**

Cellacare® Materna Comfort is a pregnancy back orthosis which stabilises the lumbar spine by supporting the abdominal and back musculature, providing relief for the back and support for the belly. Integrated stays help stabilise the lumbar spine. The stabilising effect of the orthosis can be adjusted using the integrated pulling elements.

**Product composition:**

Polyamide, polyester, cotton, polyisoprene, elastane, polyurethane, polyethylene

**Intended purpose:**

Orthosis for stabilisation of the lumbar spine during pregnancy

**Indications:**

- Non-specific pain in the lumbar spine and iliosacral region caused by incorrect loading or poor posture during pregnancy

**Contraindications:**

- Known allergy and / or hypersensitivity to any of the product components

**Side effects:**

In very rare cases, skin irritations and / or allergies may occur.

**Warnings and precautions:**

- Do not use Cellacare® Materna Comfort unless directed by a health care

professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.

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- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- Cellacare® Materna Comfort shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before use.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.
- Cellacare® Materna Comfort must be removed during longer periods of rest (e.g. when sleeping) to avoid undesirable congestive effects.
- Cellacare® Materna Comfort is intended for use on a single patient only.
- The product's durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fastener. To prevent this, the hook-and-loop fastener should always be fastened, for example when the product is not being worn or when being washed.
- Cellacare® Materna Comfort must not be used beyond the last day of the month indicated on the packaging (expiry date).
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

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**Usage information:**

**Preparation:**

1. Apply Cellacare® Materna Comfort while standing.
2. For hygienic reasons wearing a vest underneath the orthosis is recommended.
3. Open the hook-and-loop fastener and flatten out the orthosis in front of you (Fig. 1).

**Use:**

1. Apply the orthosis around the lower back and underneath the belly (Fig. 2).
2. Close the orthosis under the belly by using the hook-and-loop fastener (Fig. 3).
3. Pull the additional pulling elements, which are placed on the sides, to the front. Make use of the integrated hand grips (Fig. 4). Fasten the pulling elements in a way that a pleasant sensation of support in the back and belly area occurs (Fig. 5).
4. Check the fit of the orthosis. If required, increase or reduce the pressure by pulling harder or less hard on the additional pulling elements at the sides.

**General information:**

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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**Care recommendations:**

Machine wash at 30°C (delicates cycle) with like colours. Do not bleach. Allow Cellacare® Materna Comfort to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. Fasten the hook-and-loop closure and if possible use a mesh bag. Clean the hook-and-loop closure regularly so it retains its ability to fasten tightly, and to preserve the performance of the product.

**Storage and transport conditions:**

Cellacare® Materna Comfort must be kept away from light and moisture (preferably in the original packaging).

**Disposal:**

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

**General information:**

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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Poliamida, poliéster, algodón, poliisopreno, elastano, poliuretano y polietileno

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