medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth
Germany
T +49 921 9120
F +49 921 912780
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
 Unit 4/13-21 Hallmark Street
 Pendle Hill NSW 2145
 Australia
 T +61 2 9890 8696
 F +61 2 9890 8439
 sales@medi.au
 www.medi.australia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium NV
 Staatsbaan 77/0099
 3945 Ham
 Belgium
 T: +32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Canada Inc / médi Canada Inc
 104-1375 Lionel-Boulet,
 Varennes, Québec,
 QC Canada J3X 1P7
 T +1 450-583-3317 / +1 800 361 3153
 F +1 888-583-6827
 service@medicanada.ca
 www.medicana.ca

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

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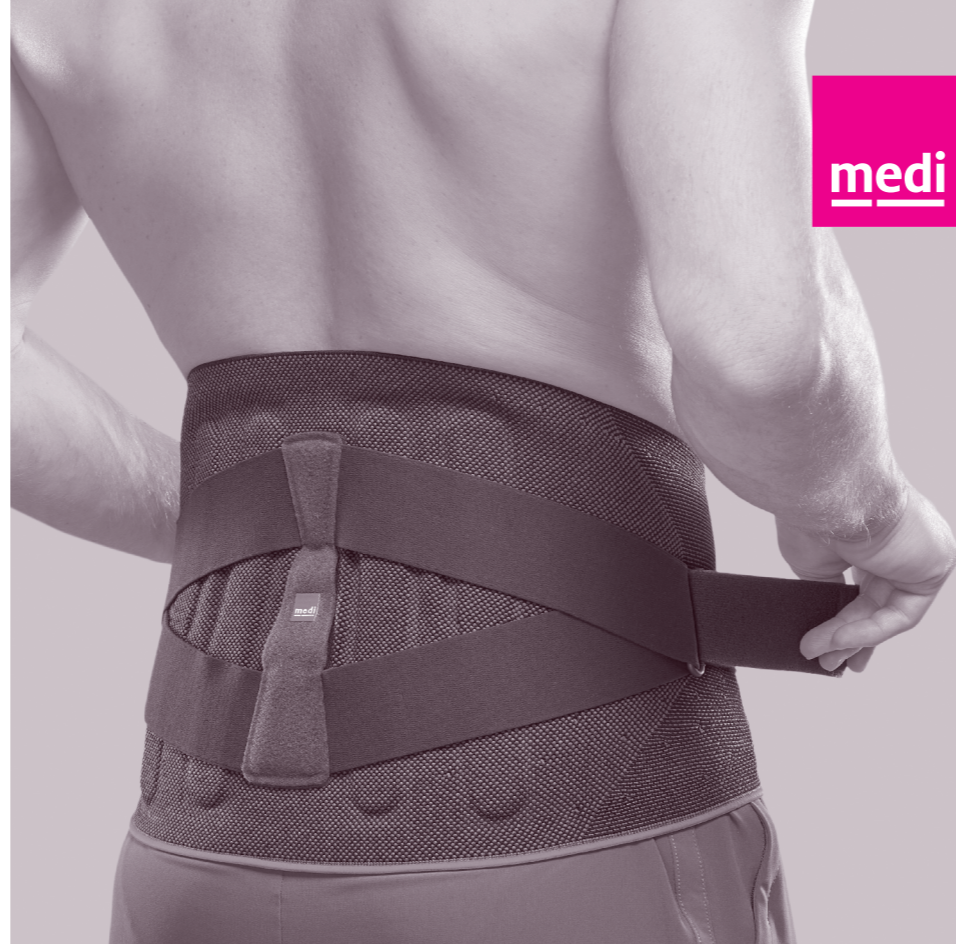
medi Trading (Shanghai) Co. Ltd.
 Room 613, HnR Group Building
 No.1178, Tian Yao Qiao Road
 Xuhui District, Shanghai, 200232
 People's Republic of China
 T +86-21 50582318
 info@medi.cn
 www.medi.cn

Medi Turk Ortopedi Medikal Ic ve Dis Tic. AS
 Mustafa Kemal Mah. 2156 Sokak No: 16 /4-5
 06510 Çankaya Ankara
 Turkey
 T: +90 312 435 20 26
 F: +90 312 434 17 67
 info@medi-turk.com
 www.medi-turk.com

UK Responsible Person
 medi UK Ltd.
 Plough Lane
 Hereford HR4 0EL
 Great Britain
 T +44-1432 37 35 00
 F +44-1432 37 35 10
 enquiries@mediuk.co.uk
 www.mediuk.co.uk

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Important notes

This medical device **MD** is made only for multiple uses on a single patient. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If you experience excessive pain or any discomfort whilst wearing, remove the product and immediately contact your doctor or medical supply store. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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Lumbamed[®] pro

Intended purpose

Lumbamed pro is a brace for partial stress relief  via correction of lordosis and functional mobilisation  of the lumbar spine.

Indications

All indications requiring partial relief by reduction of lordosis of the lumbar spine with subsequent functional mobilisation of the lumbar spine such as:

- Moderate lumbago combined with sciatica with slight deficits in the case of disc protrusion / prolapse
- Moderate pseudoradicular lumbar syndrome
- Spondylolisthesis, Grade I, with lower back pain
- Medium-grade lumbar deformity (recurrent treatment, complicated progression)
- Condition after intervertebral disc surgery with small to moderate prolapse
- Degeneration

Contraindications

None known at present.

Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paresthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally,

swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

Note on wearing

Supports have the greatest effect during physical exercise. In principal, the support can be worn all day. The special cut in the area of the groin is decisive for the correct and comfortable fit of the support. However, this depends on how comfortable the support is to wear. The support should be taken off for longer breaks (e.g. sitting for a long time or a long car drive, sleeping).

The support is suitable for use in water. After use, please rinse thoroughly.

Fitting instructions

- Before you put the brace on, loosen the straps so that the brace can be put on easily. Release the belts' fasteners and fix them loosely at the back of the belt, on the velour. Fig. 1

- Pull the fabric cover over the massage pad and position it centrally and/or according to the physician's or healthcare professional's instructions on the middle dark grey hook and loop fastener on the back element. Fig. 2
- The pad should be secured in such a way that the bottom end rests against the top of the area where the two buttocks come together.
- Use the magenta-coloured edge (bottom border) as a guide when putting on the brace.
- First, slide your left hand into the inside hand loop on the left side of the fastening. Fig. 3
- Wrap the brace around your body and slide your right hand into the outer hand loop on the right side of the fastening. Fig. 4+5
- Pull both fastening sides together in the middle so that they overlap as evenly as possible. Fig. 6
- When fastening, make sure that the back strutting is positioned in the middle. Fig. 7
- Now first release the fastening straps' fasteners and pull the straps to the front simultaneously. Fig. 8+9
- You can adjust the tightness of the brace yourself according to your needs and level of pain. The brace is positioned correctly when the bottom fastening edge is just above the pubic bone. Fig. 10
- The small finger tab on the edge of the right side of the fastener will help you to undo the brace again. Fig. 11

Note for technicians: The strap may be cut to the length required by the individual at the edge of the Y hook and loop fastener. Fig. 12

Care instructions

The pad should be removed before washing. If necessary, it can be cleaned with a dampened cloth. Please remove

the strap system before washing by releasing it from the connecting piece. Hook and loop fasteners should be closed for washing. We recommend using a laundry bag. Fabric softeners, fats, oils, lotions, ointments and soap residues can affect the material, hinder the Clima Comfort effect and cause skin irritations.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

Material composition

Knitwear: Polyamide, elastane, polyester, viscose

Pad: Polyurethane, polyethylene

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

Dispose of properly after use.

Your medi team

Wishes you a speedy recovery!

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only

serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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