medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-57
info@medi.de
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61 2 9890 8696
F +61 2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43-512 57 95 15
F +43-512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium NV
Staatsbaan 77/0099
3945 Ham
Belgium
T: +32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Canada Inc / médi Canada Inc
597, Rue Duvernay, Verchères
QC Canada J0L 2R0
T: +1 450-583-3317 / +1 800 361 3153
F: +1 888-583-6827
service@medicanada.ca
www.medicanada.ca

MAXIS a.s.,
medi group company
Slezská 2127/13
120 00 Prague 2
Czech Republic
T: +420 571 633 510
F: +420 571 616 271
info@maxis-medica.com
www.maxis-medica.com

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbaek Strand
Denmark
T +45 70 25 56 10
F +45 70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi.hu
www.medi.hu

medi Italia s.r.l.
Via Giuseppe Ghedini, 2
40069 Zola Predosa (BO)
Italy
T: +39-051 6 13 24 84
F: +39-051 6 13 29 56
info@medi-italia.it
www.medi-italia.it

medi Japan K.K.
5-1 Nihonbashi Kabutocho Chuoku
103-0026 Tokyo
Japan
T: +81 3 6778 2590
F: +81 3 5847 7901
info@medi-japan.jp
www.medi-japan.co.jp

Tachezy Sanit Srl.
Via Druso 48/a
39100 Bolzano
Italy
T: +39 0471 288 068
F: +39 0471 282 773
info@tachezysanit.com
www.tachezysanit.com

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi

medi Arm Butler

Gebrauchsanweisung. Instructions for use.
Mode d'emploi. Instrucciones de uso.
Instruções de utilização.
Gebruiksaanwijzing. Brugsanvisning.
Bruksanvisning. Instrukcja zakładania.

4 046114 701217


E006682 / 07 2021


medi. I feel better.

1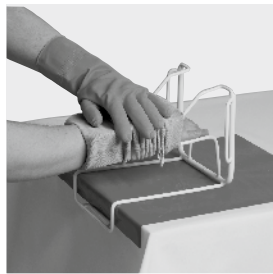


5

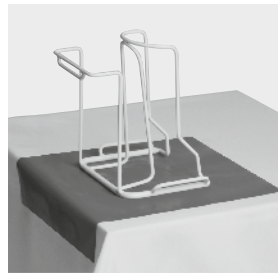2


6

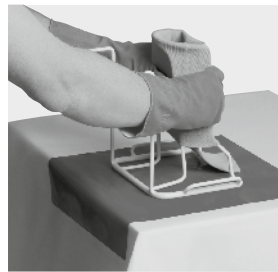3

7

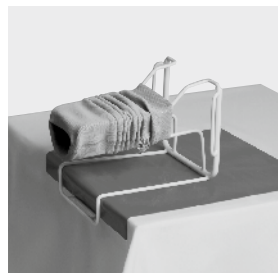4

medi.biz/an-auszieh-videos

medi.biz/donning-doffing-videos

medi Arm Butler

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medi Arm Butler

General Instructions

Please read through the entire instructions for use carefully. They contain important information concerning the product itself, as well as its handling, maintenance and disposal.

Keep the instructions for use. You may wish to read them again at a later point in time.

If you have any questions, please contact the medical supply store or the medi Consumer Service.

Compression therapy aids include, among other items, medical compression stockings which are manufactured in different lengths, compression classes and with different knitting methods. Compression stockings are used for a variety of purposes, including* treating venous disorders and conditions and for treating oedema caused by lymphoedema and lipoedema.

¹ Donning and doffing aids protect and preserve the stocking material and make it easier to put on and take off medical compression stockings. In particular, they also enable patients with limited mobility to use the products by reducing the amount of motion and force needed to put them on and take them off.

* For indications for medical compression stockings see medi and/or mediven instructions for use.

Purpose and use

Donning aid for medical/clinical compression sleeves.

Indications for donning and doffing aids include

- Hemiplegia
- Loss of power due to age

- Osteoarthritis/rheumatism
- Morbid obesity
- Extensive spinal/hip/knee stiffness
- Degenerative condition of the hands/related to the hands
- Sequellae from injuries/ amputations

¹ S2k guideline: Medical compression therapy of the extremities using medical compression stockings (MCS), venous compression bandage (VCB) and medical adaptive compression systems (MACS). Published online at: <https://www.awmf.org/leitlinien/detail/II/037-005.html> Chapter: 2.1.3 Donning and Doffing Aids

Contraindications

No contraindications are known.

Important general instructions

Ensure that your skin is smooth and unbroken and that your hands and nails are clean and well-cared for in order to avoid causing injuries or damage to the delicate fabric.

Please dispose of this product as soon as it shows signs of wear or damage.

Storage instructions

Please store the medi Arm Butler in a dry place and keep it out of direct sunlight.

How do you put on your compression sleeve using the medi Arm Butler?

Tip: Use the medi fabric gloves or rubber gloves (Fig. 1).

Place the medi Arm Butler on a table (Fig. 2).

medi tip:

Place an anti-slip surface under the medi Arm Butler (i.e. a sponge

cloth, rubberised sheet etc.)

Feed the sleeve onto the semicircular cylinder.

Please note that the seam is positioned slightly offset to the dead centre of the side (Fig. 3).

Sleeves for the left arm - slightly to the outer left (at approx. 8 o'clock).

Sleeves for the right arm - slightly to the outer right (at approx. 4 o'clock).

Now lie the medi Arm Butler flat on the table.

The bracket stabilises this donning aid on the edge of the table.

Insert your arm into the sleeve, keeping the angle the same and without turning it (Figs. 5 and 6).

Check that the sleeve is positioned correctly again (the position of the back seam, upper fastening, crook of arm etc.) and pull up the top band or the fastening if required (Fig. 7).

Tip: The medi fabric gloves and rubber gloves enable you to arrange and position compressive aids on arms and legs correctly.

Cleaning the medi Arm Butler

If desired, you can clean your donning aid with the standard commercially available disinfectants or detergents.

Please wipe down the frame and make sure that no liquid gets into the hollow part of the base (5).

Material composition

Steel

Finishing: powder-coated, white, glossy

Disposal

Please dispose of the medi donning aid correctly at the respective collection point for scrap metal.

There are no specific criteria for disposal as hazardous waste.

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