medi GmbH & Co. KG
 Medicusstraße 1
 D-95448 Bayreuth
 Germany
 T +49 921 912-0
 F +49 921 912-780
 ortho@medi.de
 www.medi.de

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium bvba
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T +32 011 24 25 60
 F +32 011 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Neuza 216
 Diadema-Sao Paulo
 Cep 09941-640
 Brazil
 T +55 11 3201 1188
 F +55 11 3201 1185
 sac@medibrasil.com
 www.medibrasil.com

medi Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbæk Strand
 Denmark
 T +45 46 55 75 69
 F +45 70 25 56 20
 kundeservice@sw.dk
 www.medit danmark.dk

medi Bayreuth España SL
 C/Canigó 2 – 6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34 93 260 04 00
 F +34 93 260 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33 1 48 61 76 10
 F +33 1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1371 0090
 F +36 1371 0091
 info@medi.hu
 www.medi.hu

medi Medical Support Sdn Bhd
 medi representative office Asia
 Unit No. B-2-19, Block B, No.2,
 Jalan PJU 1A/7A
 Oasis Ara Damansara, PJU 1A,
 47301 PETALING JAYA
 Selangor Darul Ehsan
 Malaysia
 T: +6 03 7832 3591
 F: +6 03 78323921
 info@medi-asia.com
 www.medi-asia.com

medi Middle East
 P. O. Box: 109307
 Abu Dhabi
 United Arab Emirates
 T +971 2 6429201
 F +971 2 6429070
 info@mediuae.ae
 www.mediuae.ae

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31 76 57 22 555
 F +31 76 57 22 565
 info@medi.nl
 www.medi.nl

medi Polska Sp. z o. o.
 Zygmunt Starego 26
 44-100 Gliwice
 Poland
 T +48 32 230 60 21
 F +48 32 202 87 56
 info@medi-polska.pl
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
 Rua Eugénio de Castro Rodrigues, n.º
 1700-184 Lisboa
 Portugal
 T +351 21 843 71 60
 F +351 21 847 08 33
 medi.portugal@medibayreuth.pt
 www.medi.pt

medi RUS LLC
 Business Center NEO GEO
 17 Butlerova str.
 Moscow 117676
 Russia
 T +7 (495) 374-04-56
 F +7 (495) 374-04-56
 info@medirus.ru
 www.medirus.ru

medi UK Ltd.
 Plough Lane
 Hereford HR4 0EL
 United Kingdom
 T +44 1432 37 35 00
 F +44 1432 37 35 10
 enquiries@mediuk.co.uk
 www.mediuk.co.uk

medi Ukraine LLC
 Tankova Str. 8, office 35
 Business-center "Flora Park"
 Kiev 04112
 Ukraine
 T +380 44 591 1163
 F +380 44 455 6181
 office@medi.ua
 www.medi.ua

medi USA L.P.
 6481 Franz Warner Parkway
 Whitsett, N.C. 27377-3000
 USA
 T +1 336 4 49 44 40
 F +1 888 5 70 45 54
 orders@mediusa.com
 www.mediusa.com

medi

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Important notes

This product is made for single patient use only. If the support is used for the treatment of more than one patient or is improperly modified, the product liability of the producer lapses according to the Medical Product Law. If you feel too much pain or an unpleasant feeling when wearing the support, take it off at once and contact your doctor or your specialist provider.

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Polyamid, Polyester, Elasthan, Viskose

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Genumedi®

Intended purpose

Genumedi is a knee support with patella silicone ring. The product is exclusively to be used for the orthotic fitting of the knee and only on unbroken skin or if any wounds have been properly covered.

Areas of use

- Inflammation of the joint (Arthritis)
- Painful joint wear (arthrosis)
- Irritation of the joint (e.g. excessive stress, injuries, operations) linked to swelling and effusion
- Pain in the kneecap (femoropatellar pain syndrome, anterior knee pain)

Side-Effects / Contraindications

No general side effects are known. However, if supports (or similar aids) are too tight, local pressure effects or constriction of blood vessels or nerves may occur in individuals. If you have any of the following complaints please consult your doctor before using the brace:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).
- Sensitivity and perfusion disorders of the legs or feet (e.g. in the case of diabetes).
- Lymph outflow disorders, as well as undiagnosed swellings of soft tissues, apart from the area of use.

Any combination with other products (e.g. compression stockings) must be discussed in advance with the responsible doctor.

Mode of Action / Comment on Wear

Please take care that trained staff instruct you when putting on the support for the first time.

The integrated silicone ring of the Genumedi reduces stress on the kneecap. Together with the compression of the knitted fabric, it causes a massage effect. This enhances perfusion of the joint and decreases swelling and effusion. Irritation and pain are alleviated. At the same time, muscular control and inherent stabilisation of the knee joint are improved.

Supports are most beneficial during physical activity. The support can in principle be worn throughout the day. This must depend on how comfortable the wearer is. Moreover, the knee may not be bent by more than 70° when sitting. The support should be removed for long breaks (e.g. long periods sitting, long drives or sleeping).

If you suffer excessive pain or discomfort during wear, please take off the product immediately and contact your doctor or specialised shop.

The support is suitable for use in water. After use, please rinse thoroughly

Putting on / Taking off

- Grasp the support with the thumb on the internal upper end of the lateral knob adhesive bands (Fig. 1)
- Pull the support over the knee in such a way that the silicone ring covers the kneecap centrally and free of pressure (Figs. 2 +3).
- To take off the support, grasp the lower lateral rods and pull it downwards and away (Fig. 4).

Care instructions

The silicone ring does not need to be removed before washing. Fabric softener, fats, oils and ointments can attack the material and the Clima Comfort Effect can be impaired. Soap residues can cause skin irritation and material wear.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

Material composition

Polyamide, polyester, elastane, viscose

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

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