medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T +32 011 24 25 60
F +32 011 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
Rua Itapeva 538 cj. 42
Bela Vista, São Paulo - SP
CEP: 01332-000
Brasil
Tel: +55 (11) 3500 8000
sac@medibrasil.com
www.medibrasil.com

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbæk Strand
Denmark
T +45 70 25 56 10
F +45 70 25 56 20
kundeservice@sw.dk
www.meditandmark.dk

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi.hu
www.medi.hu

medi Medical Support Sdn Bhd
medi representative office Asia
Unit No. B-2-19, Block B, No.2,
Jalan PJU 1A/7A
Oasis Ara Damansara, PJU 1A,
47301 PETALING JAYA
Selangor Darul Ehsan
Malaysia
T: +6 03 7832 3591
F: +6 03 7832 3921
info@medi-asia.com
www.medi-asia.com

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Norway AS
P.B 238
4291 Kopervik
Norway
T +47 528 44 50 0
F +47 528 44 50 9
post@medinorway.no
www.medinorway.no

medi Orient M.E.A. FZC
TAURUS GROUP
Suite No. M01 Wasel Al Mamzar Bldg.
Al Wuhida Road, Al Mamzar
P.O.BOX NO. 91464
DUBAI, UAE
T +971 4 2557344
F +971 4 2557399
md@taurusgroup.net

medi Polska Sp. z. o. o.
Zygmunt Starego 26
44-100 Gliwice
Poland
T +48 32 230 60 21
F +48 32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi RUS LLC
Business Center NEO GEO
Butlerova st., 17
117342 Moscow
Russia
T +7 (495) 374-04-56
F +7 (495) 374-04-56
info@medirus.ru
www.medirus.ru

Lumbamed® basic

Lumbalorthese Lumbar support Ceinture des soutien lombaire Soporte lumbar

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso. Instruções para aplicação. Istruzioni per l'uso. Gebruiksaanwijzing. Brugsvejledning. Bruksanvisning. Návod k použití. Uputa za uporabu. Инструкция по использованию. Kullanna kilavuzu. Instrukcja zakładania. Οδηγία εφαρμογής. Használati útmutató. Uputstvo za upotrebu. Інструкція з використання. Navodila za uporabo. 使用说明书.

medi

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Important notes
This product is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the support, please consult your doctor or orthotist immediately. Do not wear the brace over open wounds, and use it only as instructed by your doctor or orthotist.

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Polyamid, Polyester, Elasthan, Viskose

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Lumbamed[®] basic

Intended purpose

The Lumbamed basic is a lumbar support. The product is exclusively to be used for the orthotic fitting of the lower back and only on intact skin.

Areas of use

- Non-specific pain in the area of the lumbar spine
- Wear-related pain in the area of the lumbar spine
- Irritation of the sacro-iliac joint
- Excessive stress on the border between the lumbar spine and the pelvis
- Weak or imbalanced musculature in the trunk (back or stomach)
- Loosening of the pubic symphysis

Side-Effects / Contraindications

There are no known side effects if the correctly sized product is fitted as recommended. If unexpected pain, swelling or numbness occurs while wearing the support, remove it and seek medical advice. Particular care is required where the following conditions are present:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).

Any combination with other products must be discussed in advance with the responsible doctor.

Mode of Action/Comment on Wear

Please note that initial fitting should be supervised by a healthcare professional. With the combination of the compressive knitted fabric and the integrated back rods, Lumbamed basic

stabilises the area of the lumbar spine and the border to the pelvis, without impairing muscular strength. The static forces in the vertebral column are improved and stress on the affected segments is reduced. This alleviates pain.

Supports cause the greatest effect during physical exercise. In principal, the support can be worn all day. However, this depends on how comfortable the support is to wear. The support should be taken off for longer breaks (e.g. sitting for a long time or a long car drive, sleeping).

If you suffer excessive pain or discomfort during wear, please take off the product immediately and seek medical advice.

The support is also suitable for use in water. After use, please rinse out the product thoroughly.

Putting on / taking off

- The label on the right inside of the fastener explains how to put the support on (Fig. 1).
- First, slip into the wrist strap in the left inside of the fastener. Wrap the bandage around the body and then slip into the outer wrist strap on the right side (Fig. 2+3).
- Pull the two fastening sides over each other and then secure them as centrally as possible with the Velcro. The bandage has been correctly applied when the bottom edge of the fastener is located just above the pubic bone (Fig. 4+5).
- The small finger tab on the edge of the right side of the fastener will help you to undo the bandage again (Fig. 6).

The Clima Comfort knitted fabric will achieve its best breathing activity when the bandage is worn directly against the skin.

Care instructions

Velcro fasteners should be closed for washing. Fabric softener, fats, oils and ointments can attack the material and the Clima Comfort Effect can be impaired. Soap residues can cause skin irritation and material wear.

- You can wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

Material composition

Polyamide, polyester, elastane, viscose

Warranty / contractual guarantee

Consistent with legal guidelines.

Disposal

The product can be disposed of in the domestic waste.

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