medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-783
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T + 32-11 24 25 60
F +32-11 24 25 64
info@medibelgium.be
www.mediibelgium.be

medi Brasil
Rua Itapeva 538 cj. 42
Bela Vista, São Paulo - SP
CEP: 01332-000
Brazil
T +55-11-3500 8005
sac@medibrasil.com
www.mediBrasil.com

medi Danmark ApS
Vejlegårdsvej 59
2665 Vallensbaek Strand
Denmark
T +45-70 25 56 10
F +45-70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Bayreuth Espana SL
C/Canigo 2-6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34-932 60 04 00
F +34-932 60 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33-1 48 61 76 10
F +33-1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1 371-0090
F +36 1 371-0091
info@medi.hu
www.medi.hu

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31-76 57 22 555
F +31-76 57 22 565
info@medi.nl
www.medi.nl

medi Orient M.E.A. FZC
TAURUS GROUP
Suite No. M01 Wasel Al Mamzar Bldg.
Al Wuhida Road, Al Mamzar
P.O.BOX NO. 91464
DUBAI, UAE
T: +971 4 2557344
F: +971 4 2557399
md@taurusgroup.net

medi Polska Sp. z o.o.
Zygmunt Starego 26
44-100 Gliwice
Poland
T +48-32 230 60 21
F +48-32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
Rua do Centro Cultural, no. 43
1700-106 Lisbon
Portugal
T +351-21 843 71 60
F +351-21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LLC
Business Center NEO GEO
Butlerova Street 17
117342 Moscow
Russia
T +7-495 374 04 56
F +7-495 374 04 56
info@medirus.ru
www.medirus.ru

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
Great Britain
T +44-1432 37 35 00
F +44-1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Ukraine LLC
Tankova Str. 8, office 35
Business-center „Flora Park“
Kiev 04112
Ukraine
T +380 44 591 11 63
F +380 44 392 73 73
info@medi.ua
http://medi.ua

medi USA L.P.
6481 Franz Warner Parkway
Whitsett, N.C. 27377-3000
USA
T +1-336 4 49 44 40
F +1-888 5 70 45 54
info@mediusa.com
www.mediusa.com

Lumbamed® plus

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Important notes

This product is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the support, please consult your doctor or orthotist immediately. Do not wear the brace over open wounds, and use it only as instructed by your doctor or orthotist.

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Bandage: Polyamid, Polyester,
Elasthan, Viskose

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Lumbamed[®] plus

Intended purpose

The Lumbamed plus is a lumbar support with pad. The product is exclusively to be used for the orthotic fitting of the lower back and only on intact skin.

Areas of use

- Non-specific pain in the area of the lumbar spine (LS)
- (Repeated) Pain due to attrition in the area of the LS, e.g. on account of changes to the intervertebral disks
- Irritations of the sacroiliac joint
- Overstrain injuries in the transitional area lumbar spine/pelvis
- Weak or unbalanced trunk muscles (back, abdomen)

Side-Effects / Contraindications

No side-effect is known which influences the whole body. However, if supports (or similar aids) are too tight, this may lead to local pressure effects or the constriction of blood vessels or nerves in individual cases. You should thus consult your doctor in the following circumstances before applying the support:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).

Any combination with other products must be discussed in advance with the responsible doctor.

Mode of action / Comment on wear

Please note that initial fitting should be supervised by a healthcare professional.

Lumbamed plus has a stabilising and relieving effect on the lumbar spine. Its compressive knitted fabric and the pad

that is used also stimulate circulation and have a pain-relieving, massage effect without affecting the muscular force.

Supports have the greatest effect during physical exercise. In principal, the support can be worn all day. The special cut in the area of the groin (Comfort Zone) is decisive for the correct and comfortable fit of the support. However, this depends on how comfortable the support is to wear. The support should be taken off for longer breaks (e.g. sitting for a long time or a long car drive, sleeping).

If you suffer excessive pain or discomfort during wear, please take off the product immediately and contact your doctor or specialized shop.

The support is also suitable for use in water. After use, please rinse out the product thoroughly.

Putting on / Taking off

- Pull the fabric cover over the massage pad and secure the pad in the centre or in accordance with the physician's or healthcare professional's instructions on the middle grey Velcro section between the two rods. The pad should be secured in such a way that the bottom end rests against the top of the area where the two buttocks come together (Fig. 1+2).
- The label on the right inside of the fastener provides information about how to put the support on (Fig. 3).
- First, slip into the wrist strap in the left inside of the fastener (Fig. 4).
 - Wrap the bandage around the body and then please refer back to the label to check whether you are holding the bandage correctly (Fig. 5).
 - Now, slip into the outer wrist strap on the right side (Fig. 6).

- Pull the two fastening sides over each other and then secure them as centrally as possible with the Velcro(R). The bandage has been correctly applied when the bottom edge of the fastener is located just above the pubic bone (Fig. 7+8).
- The small finger tab on the edge of the right side of the fastener will help you to undo the bandage again (Fig. 9).

The Clima Comfort knitted fabric will achieve its best breathing activity when the bandage is worn directly against the skin.

Care instructions

Do not wash the pad together with the support. It can be cleaned with a damp cloth. Velcro fasteners should be closed for washing. Fabric softener, fats, oils and ointments can attack the material and the Clima Comfort Effect can be impaired. Soap residues can cause skin irritation and material wear.

- You can wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

Composition of the Material

Support: Polyamide, polyester, elastane, viscose

Pad: Polyurethane, polyethylene

Warranty / contractual guarantee

Consistent with legal guidelines.

Disposal

The product can be disposed of in the domestic waste.

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