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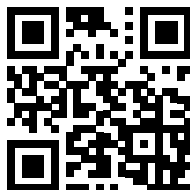
Instructions for use

ofa[®]

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english

Fitting instructions

The pictures that go with the fitting instructions can be found on the inside cover (2) at the start of the brochure.

Note on the hygiene cover:

the hygiene cover included with the Walker is intended for indoor use only and protects upholstered furniture and beds.

1 Put on the Walker when sitting down. If necessary, put the stocking that is provided on. To put the Walker on, open all of the Velcro fasteners and the fabric cuff. Place your leg in the cuff with your heel against the back of the Walker.

2 First close the Velcro fastener on the cuff over your instep and then close the toe flap. Then close the cuff on the shin. Put the plastic splint on the front by fastening it to the cuff using Velcro.

3 Then close the Velcro straps from the bottom to the top by inserting them through the eyelet provided and fastening them tightly. Ensure that the product is secure but not too tight so as to prevent pinching. The Walker must feel comfortable.

4 Finally, adjust the air cushions to the desired pressure. Insert the air pump provided with the light tip ("In") into the blue valve and inflate the air cushions to the desired level. Do not overinflate the cushions. The air should be let out before you take the Walker off by placing the black tip of the pump ("Out") into the valve and pumping the air out. This makes the Walker easier to take off and put back on and avoids overinflating the air cushions.

Fitting the optional wedge set (only by experts)

Certain clinical pictures require the foot to be stabilised with the toes pointed (plantar flexion). A wedge set can be requested for the high version of the

Walker for this purpose and is inserted into the cuff of the Walker. The wedge set individually tailored to you by trained experts. The set consists of five wedges joined together by means of adhesive points. Wedges 1 to 3 should be used for size S, wedges 2 to 4 for size M and wedges 3 to 5 for sizes L and XL. Three wedges one on top of the other correspond to a toe pointed at an angle of approximately 28°.

Intended Use

The product is designed to immobilise the ankle joint and the foot. A solid plastic shell limits the degrees of freedom of the ankle joint and the foot.

⚠ Important Information

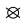
- The initial fitting and adjustment of the product as well as instructions on its proper and safe use should be provided by trained medical/orthopaedic specialists.
- The product should only be used for the indications in the adjacent column.
- Ensure that the product fits correctly and check it before each use for functionality, wear and damage (e.g. breaks, tears or deformations). A damaged product may no longer be worn.
- This product may only be worn by persons weighing less than 125 kilograms.
- Wear the product directly on the skin. It should only be worn over intact skin. If it is to be worn in combination with other products, please discuss this with your doctor first.
- If you experience an unpleasant sensation, severe pain or other symptoms while wearing the product, remove it and contact your doctor or specialist supplier.
- We use a comprehensive quality assurance system to check our products. If you have any complaints, however, please contact your specialist supplier.

Instructions for care

- The product should be washed before its first use and then on a regular basis.
- Detach the removable padding from the shell for washing. Ensure that you do not pull the Velcro strips off the splints too. The plastic shell can be cleaned with a damp cloth.

- Wash the fabric parts by hand at a maximum of 30 °C. Use a mild detergent that does not contain fabric softener (e.g. Ofa Clean special detergent) and rinse the product out well. Squeeze the water out carefully without wringing the product.
- Mould the product into the right shape and let it dry in the air.
- To maintain the quality of your product, please do not use any chemical cleaning products, bleaches, petrol or fabric softener on it. Do not apply any fatty or acidic creams, ointments or lotions to the skin in the area where the product is used. These substances may damage the material.
- Store the product in a dry place and protect it from sunlight and heat, preferably in the original packaging.

Care

-  Hand wash
-  Do not bleach
-  Do not tumble dry
-  Do not iron
-  Do not dry clean

Do not use fabric softener!

Reporting obligation

In accordance with legal provisions within the EU, patients and users are obliged to report any serious incident that occurs when using a medical device to both the manufacturer and the competent national authorities (in Germany BfArM, Bundesinstitut für Arzneimittel und Medizinprodukte, the Federal Institute for Drugs and Medical Devices) immediately.

Indications

- Severe ankle sprains
- Stable foot or ankle fractures
- Immobilisation before and after operations
- Torn Achilles tendons (only the high version of the Walker)
- Treatment-resistant Achilles tendinitis (only the high version of the Walker)

Side effects

There are currently no known side effects for any part of the body if the product is used correctly.

However, if the product is too tight it can cause local pressure points or constrict blood vessels and nerves.

Contraindications

Consult your doctor before using this product in the following cases:

- Skin diseases or injuries in the area where the product is used, particularly if you experience any signs of inflammation such as redness, an increase in temperature or swelling
- Feeling of numbness and circulatory problems in the area where the product is used
- Problems with lymphatic drainage and unexplained swelling

Material Composition

Insole:	76 % Polyurethane
	13 % Polyester
	11 % Polyamide
Walker:	85 % Polypropylene
	15 % Glass Fibre
Wedges:	100 % EVA
Air cushions:	100 % Polyurethane

Disposal

Please dispose of the product according to local regulations after the end of use.

Guarantee

If used as intended and maintained properly, the product can be used for six months. The product should be handled carefully in line with these instructions for use. Improper use or improper changes to the product may negatively impact the performance, safety and functionality of the product and lead to the exclusion of claims. The product is designed for the treatment of a single patient. It is not designed to be reused by a second patient or any further patients.

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Ofa Bamberg GmbH