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INSTRUCTIONS FOR USE EN

INTENDED USE

OMO-HiT® ABDUCTION is a shoulder joint brace for immobilisation and relief of the shoulder joint, by which the arm is brought into a required abduction position in the shoulder joint.

INDICATIONS

- Subacromial bursitis / impingement
- Mobilisation after frozen shoulder
- Pre- and postoperative after trauma and/or joint replacement
- For conservative treatment of injuries to the shoulder-arm apparatus, especially injuries/ruptures of the rotator cuff
- All conditions requiring fixation of the arm in abduction position

CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage).

SIDE-EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

PUTTING IT ON AND USE

- Open the quick release of the shoulder strap. Support the injured arm and slide the forearm into the sling. Place the adjustable shoulder strap around the neck and close the quick release **1**.
- Both ends of the shoulder belt are adjustable. Adjust the shoulder strap to the required length **2**.
- The shoulder pad is velcroed in and can be adjusted. To do this, remove the pad from the shoulder strap and reposition it individually **3A**.
- Attach the closing strap to the top of the sling near the elbow **3B**.
- Attach the thumb strap to the open end of the sling between the thumb and fingers **3C**.
- Place the abduction cushion on the inside of the brace. With the abduction cushion you can achieve an abduction angle of approx. 30°. This depends on your body size and soft tissue situation **4**.

- Place the strap of the abduction cushion around your waist and close it. Adjust the strap to the desired length **5**.
- Attach the ball for hand positioning and training to the long side of the abduction cushion **6**.

NOTES FOR THE ORTHOPAEDIC TECHNICIAN

- If the length is too long, the straps can be shortened. To do this, first remove the tab at the end of the shoulder strap and/or the abdominal strap **7A**.
- Cut the straps to the desired length with scissors **7B**. Then place the straps as described above.

⚠ IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

CARE

We recommend washing OMO-HiT® ABDUCTION gently in lukewarm water (30°C) with mild detergent, rinse thoroughly and air dry. Do not dry the product in the tumble dryer, on the heating or in the microwave. The abduction pad can be wiped with a damp cloth. It must not be immersed in liquids.

TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

An online version of the instructions for use can be found on our website at www.sporlastic.de.

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