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INSTRUCTIONS FOR USE EN

INTENDED USE

OMO-HiT® IMMOBIL/OMO-HiT® SUPPORT is a shoulder joint brace for immobilisation of the shoulder joint in a defined position.

OMO-HiT® IMMOBIL has an additional circular abdominal belt for fixation of the entire arm in a defined position.

INDICATIONS

- In case of insufficiency or after injuries of the capsule-ligament apparatus of the shoulder.
- Paralysis of the arm
- Subcapital humerus fracture
- Post-traumatic complaints
- Indications requiring immobilisation of the shoulder and/or elbow joint

CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage).

SIDE-EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

PUTTING IT ON AND USE

OMO-HiT® IMMOBIL:

- Open the quick release of the shoulder strap. Support the injured arm and slide the forearm into the sling. Place the adjustable shoulder strap around the neck and close the quick-release **1**.
- Both ends of the shoulder belt are adjustable. Adjust the shoulder strap to the required length **2**.
- The shoulder pad is velcroed in and can be adjusted. To do this, remove the pad from the shoulder strap and reposition it individually **3A**.
- Attach the closing strap to the top of the sling near the elbow **3B**.
- Attach the thumb strap to the open end of the sling between the thumb and fingers **3C**.
- Put the body belt around the back and close the quick release **4**.
- The two ends of the body belt are adjustable. Adjust the body belt to the required length **5**.
- This illustration shows the body belt properly attached **6**.

OMO-HiT® SUPPORT:

- Open the quick-release buckle of the shoulder belt. Support the injured arm and slide the forearm into the sling. Place the adjustable shoulder strap around the neck and close the quick release **1**.
- Both ends of the shoulder belt are adjustable. Adjust the shoulder strap to the required length **2**.
- The shoulder pad is velcroed in and can be adjusted. To do this, remove the pad from the shoulder strap and reposition it individually **3A**.
- Attach the closing strap to the top of the sling near the elbow **3B**.
- Attach the thumb strap to the open end of the sling between the thumb and fingers **3C**.

NOTES FOR THE ORTHOPAEDIC TECHNICIAN

- If the straps are too long, they can be shortened. To do this, first remove the tab at the end of the shoulder strap and/or the waist strap **7A**.
- Cut the straps to the desired length with scissors **7B**. Then place the straps as described above.

⚠ IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

CARE

We recommend washing OMO-HiT® IMMOBIL/ OMO-HiT® SUPPORT gently in lukewarm water (30°C) with mild detergent, rinse thoroughly and air dry. Do not dry the product in the tumble dryer, on the heating or in the microwave.

TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

An online version of the instructions for use can be found on our website at www.sporlastic.de.

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