TOILETTENSTUHL RFM[®] FAHRBAR

Gebrauchsanweisung | Instructions for use | Mode d'emploi | Istruzioni per l'uso |
Instrucciones de uso | Gebruiksaanwijzing | Brugsanvisning

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Material Rad Polyvinylchlorid (PVC), Thermopl. Gummi (TPR)

Farbe anthrazit / schwarz

INSTRUCTIONS FOR USE COMMODE CHAIR RFM® MOBILE

01 IMPORTANT INFORMATION

General information	10
Product labelling	10
Intended purpose and product description	10
Indication	10
Contraindication	10
Declaration of conformity	10

02 PRODUCT CONSTRUCTION

Scope of delivery	11
Structure of the commode chair RFM® mobile	11

03 PRODUCT IN OPERATION

Assembly	12
Using the brakes and parking brakes	12

04 SAFETY INSTRUCTIONS

Safety instructions	13
---------------------------	----

05 INSTRUCTIONS FOR USE

Cleaning and disinfection	13
Storage	13
Maintenance	13
Re-use	13
Disposal	13

06 MANUFACTURER'S DECLARATIONS

Service life	14
Warranty	14
Notification / Incident	14

07 TECHNICAL DATA

Dimensions and technical information	14
--	----

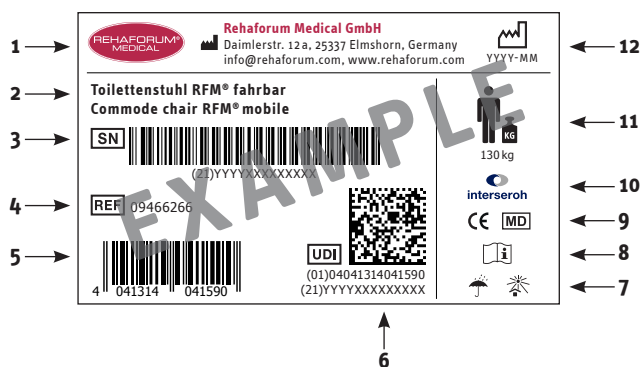
GENERAL INFORMATION

Thank you for choosing a product from Rehaforum.

Please read the complete instructions for use carefully before installing, adjusting or using the product for the first time. This will provide you with important information that will allow you to fully utilise the technical advantages. In addition, you will also receive information that will help you to ensure operational safety and maximise the value of your product. If you require further information about the product, please contact your specialist dealer.

These instructions for use contain all the necessary information for handling the product. Please keep them in a safe place so that they are available for future reference.

PRODUCT LABELLING

- 1. manufacturer | 2. article description | 3. serial number | 4. article number | 5. EAN code | 6. UDI = Unique Device Identifier, (01) UDI-DI/GTIN, (21) serial number | 7. rain protection symbol / sun protection symbol | 8. observe instructions for use | 9. CE marking / medical device | 10. dual system | 11. max. load | 12. production date

INTENDED PURPOSE AND PRODUCT DESCRIPTION

The commode chair RFM® is used for hygienic toilet use and to facilitate personal care for persons who are in need of care and/or have limited mobility.

INDICATION

The use of the product is suitable for persons with

- slight loss of balance
- slight loss of strength
- limited mobility of the legs, knee joints and upper limbs

CONTRAINDICATION

The use of the product is not suitable for persons with

- severe sensory disturbances
- severe loss of balance
- severe loss of strength
- severe pain in the trunk or in the upper and lower limbs

DECLARATION OF CONFORMITY

Rehaforum Medical GmbH declares the conformity of this product in accordance with EU Regulation 2017/745.

SCOPE OF DELIVERY

The following components and accessories are included in the scope of delivery

- Commode chair RFM® mobile
- Bucket with lid
- Seat cushion cover
- Instructions for use

STRUCTURE OF THE COMMODE CHAIR RFM® MOBILE

- | | | |
|------------------|--------------------------------|--|
| 1. Rack | 8. Seat cushion cover | 14. Footrest lock |
| 2. Frame | 9. Seat frame | 15. Pin for connecting tube of footrests |
| 3. Push bar | 10. Bucket with lid | 16. Connecting tube of footrests |
| 4. Backrest | 11. Guide rail for bucket | 17. Heel protection belt |
| 5. Locking screw | 12. Wheels without brake lever | 18. Swivelling and removable footrest |
| 6. Armrests | 13. Wheels with brake lever | |
| 7. Armrest lock | | |

ASSEMBLY


Ask your specialist dealer / care staff for assistance with installation. The commode chair RFM® mobile is supplied partially pre-assembled in a cardboard box. Check the box and the contents for integrity and completeness. **You will receive the following individual parts**

1 x pre-assembled rack | 1 x backrest | 2 x large locking screw | 2 x wheel with brake lever | 2 x wheel without brake lever | 4 x screw | 4 x nut | 1 x footrest left | 1 x footrest right | 1 x heel protection strap | 1 x bucket with lid | 1 x seat with cut-out | 1 x seat cushion cover

1. Remove the bucket and the lid (10.) from the frame (2.). Now place the rack upside down on the level floor so that the tube openings of the rack (1.) are facing upwards.

2. Insert one of the four wheels (12./13.) into each tube opening and secure each one with a screw and a nut. Ensure that the two wheels with the brake levers (13.) are inserted into the two rear tubes.

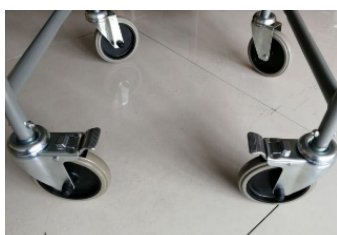3. Now turn the rack over and fix the brakes of the rear wheels (13.) by pressing down the brake levers. Insert the connecting tubes of the footrests (16.) with the overlapping openings from above into the pins (15.) provided for this purpose, which are located on the right and left of the rack. Fold the connecting tubes of the footrests inwards. The footrest locking mechanism (14.) locks them in place and they can be released again by operating the levers. Attach the heel protection strap (17.) to the connecting tubes of the footrests (16.).

4. Slide the back section (4.) from above into the tube openings provided and secure it with the two large locking screws (5.).

5. Bring the armrests (6.) into a horizontal position and secure them with the armrest lock (7.) by pulling them backwards and carefully pushing them into the openings. To fold the armrests down if necessary, pull the locking mechanism until it releases again.

6. Now attach the seat frame (9.) with the recess. To do this, first place it on the rear tube of the rack (1.). Make sure that the wide opening of the guide rail (11.) underneath the seat is pointing backwards. Fold the seat down so that it sits firmly on the rest of the rack.

7. Slide the bucket with the lid into the guide rail from behind. The bucket is held in place by two resistors in the guide rail. To empty the bucket, remove it at the same time as the lid.

8. To use the commode chair RFM® as a seat, place the seat cushion cover (8.) on the seat cushion. Always ensure that the brakes are locked before sitting on the commode chair RFM®.

USING THE BRAKES AND PARKING BRAKES

The parking brakes are located on the two rear wheels and are locked by pressing down the larger lever. The brakes are released again by pressing down the smaller lever. The directional locks are also located on the rear wheels and are activated by locking the brakes to prevent the commode chair RFM® from sliding away.

SAFETY INSTRUCTIONS

- Check that the commode chair RFM® is undamaged before each use. Do not use a defective commode chair RFM®!
- Damaged or worn parts may only be replaced by a specialist dealer.
- Ensure that the screws / tensioning screws are always tightened.
- Be careful when loosening the footrests. There is a risk of jamming!
- When pushing, make sure that the patient has their arms on the armrests and their feet on the footrests to avoid injury when pushing.
- Make sure that the patient has not placed their arms on the backrests before you release them to fold them away.
- It is forbidden to use the armrests as a seat!
- Ensure that the brakes are locked before the commode chair RFM® is used.
- Before each use, check that the seat is secure and that the cushion is not tightly fitted.
- Before each use, make sure that the bucket is positioned below the opening of the seat and that the lid has been removed.
- Avoid leaning out too far and jerky movements when using the commode chair RFM®.
- Never stand on the footrests when getting in and / or out of the commode chair RFM®.
- The commode chair RFM® is intended for indoor use only.
- The commode chair RFM® is not a wheelchair and is only suitable for short distances.
- Only use the commode chair RFM® on a level, horizontal surface.
- Before storage, the commode chair RFM® including bucket and lid must be prepared in accordance with the instructions for cleaning and disinfection.
- Do not allow children to play with the commode chair RFM®. It is not a toy!
- The maximum load capacity is 130 kg.

CLEANING AND DISINFECTION

The product should be cleaned and disinfected regularly. This is essential if the product is used / re-used by another person. To clean the product, rinse it with warm water and use a mild soapy solution for cleaning. Dry the product with a soft cloth and ensure good ventilation. Disinfection can be carried out using a commercially available disinfectant. Follow the manufacturer's instructions for use. Do not use abrasive or corrosive cleaning agents or high-pressure or steam cleaners!

STORAGE

Store the product in a dry place protected from direct sunlight. Protect the product from dust and strong heat sources. Ensure good ventilation. The product must be cleaned and dried before storage.

MAINTENANCE

If you notice any damage, signs of wear and / or loosening of the mounted parts, contact your specialist dealer.

RE-USE

The product is suitable for re-use. Before passing the product on to another user, it must be checked, cleaned, disinfected and serviced by a specialist dealer. If parts of the product are damaged or destroyed, they must be replaced with new ones. All loosened screws must be tightened. The product must be in perfect condition before being used again.

 **It is forbidden to re-use the product if irreparable damage to the product is detected! The instructions for use must be handed over with every change of user.**

DISPOSAL

Disposal options may vary depending on local regulations. Check with your local waste disposal centre whether the product can be disposed of as normal household waste or whether there are separate regulations for disposal.

SERVICE LIFE

If the product is used for its intended purpose and is cared for, checked and maintained in accordance with the instructions, a service life of five years can be assumed. However, the service life can vary considerably depending on the handling and care of the product. For example, particularly careful handling of the product can extend the service life, while extreme and improper handling of the product can shorten the service life. The determination of the service life by Rehaforum Medical GmbH does not constitute an additional guarantee.

WARRANTY

The warranty period is 24 months and applies to all product defects that are demonstrably attributable to manufacturing or material defects. Excluded from the warranty are Damage caused by natural wear and tear or intent. Furthermore, the warranty does not cover damage caused by incorrect cleaning and care products or damage caused by negligent and improper operation or use.

NOTIFICATION / INCIDENT

All serious incidents occurring in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and / or patient is established.

DIMENSIONS AND TECHNICAL INFORMATION

Item no. 09466266
HMV 18.46.02.0061
Weight 12.5 kg
Total width 55 cm
Total height 98 cm
Total depth 84 cm
Seat width 45 cm
Seat height 55.5 cm
Seat depth 44 cm
Backrest width 44 cm
Backrest height 39.5 cm
Backrest length 36 cm
Upholstery of the backrest 0.15 cm
Width between the armrests 45 cm
Armrests height 78 cm
Upholstery of the armrests 0.25 cm
Seat angle 90° with closed cover
Diameter of the wheels 12 cm / 5"
Max. load 130 kg
Material frame steel
Material seat / backrest polyvinyl chloride (PVC), foam
Material armrest / bucket incl. lid / footrest polyvinyl chloride (PVC)
Material wheel polyvinyl chloride (PVC), thermoplastic rubber (TPR)
Colour anthracite / black

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