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Aluminium-Duschklapsitz

Art.-Nr. 78048 000

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Folding Aluminium-Shower Seat

Item No. 78048 000

Please read these notes carefully before use. Be sure to follow the instructions and store the manual carefully! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

The foldable aluminium shower seat provides individuals that suffer from impaired standing stability but with sufficient sitting stability support during their daily hygiene under the shower. The recess in the seat facilitates intimate hygiene and avoids the necessity of standing up and/or bending over during bodily hygiene. The shower seat promotes safety and independence. The seat can be folded up onto the wall when not in use.

Safety Indications

- Users heavier than 130 kg may not use the folding shower chair!
- The folding shower seat must be firmly mounted on a wall in the shower area! This wall must be suitable for bearing persons weighing up to 130 kg body weight on the seat! We recommend having an expert determine the suitability of the wall and carry out the mounting!
- The folding shower seat should only be used, if the individual can sit by himself without any aid. It may not be used, if you are unable to control your sitting balance!
- Only use the folding shower seat, if it is in perfect technical condition!
- Make sure that the shower folding seat is completely unfolded and that the feet are secure each time before using - otherwise there is a risk of collapse when sitting down!
- There is a bruising hazard during unfolding and folding the seat! There is also a bruising hazard for fingers and hand when grasping through the opening between the frame and the seat!
- Do not lean too much forward when sitting on the shower seat!
- Do not alter the construction, if not approved by the manufacturer!
- Please observe that in wet shower cabins there is a sliding hazard during sitting down or getting up from the chair - in particular when using cleansing additives or soap!
- Only use the folding shower seat for its designated use and not as a stepladder substitute or toy!
- Due to the tilting hazard children may not use the shower chair unattended!

Scope

The contents of delivery includes 1 folding shower seat, 1 user manual, 1 packaging and transport box. Please check the content of the box. In case of visible faults or missing parts, please contact your specialist dealer.

Assembly/Use

The mounting should only be carried out by specialists!

- Check that the scope of supply is complete.
- Make sure that the wall is suitable for mounting.
- Chose fastening material corresponding to the nature of the wall. The screws should be made of rustproof material and must not exceed a diameter of 6 mm.
- Open out wall brackets and fold open chair legs.
- Determine the height of the mounting and the corresponding seat height adjustment. Please read the section »Adjusting the Seat Height« for this.
- Mark drill holes in the wall, drill holes in the wall corresponding to the diameter of the screws and provide with dowels.
- Align wall brackets corresponding to the drill holes and fasten screws (but not fully tight).
Attention: The wall plate should still be able to move a little!
- Align wall brackets with the spirit level and tighten screws.
- Fold out seat and align horizontally.

Components:

1. Wall bracket with fastening holes
2. Seat with water drainage holes
3. Folding bar
4. Chair legs
5. Spring pin
6. Non-slip chair leg buffer
7. Finely adjustable foot

Folding the shower seat down

Please be aware of the risk of trapping your fingers when folding the seat down!

1. Fold the shower seat down on the seating area.
2. The chair legs unfold independently at the same time. But always ensure that both feet are secure and firmly on the ground before you sit down!

Folding the shower seat up

Please be aware of the risk of trapping your fingers when folding the seat up!

1. Pull the seat up.
2. The chair legs fold in independently and the seat can be kept on the wall to save space.

Adjustment of seat height

The shower seat can be height adjusted via 4 bores in the chair legs.

1. Remove the spring pins at the chair legs.
2. Adjust desired length by pulling out or shifting in the chair legs. Ascertain that the bores of the chair legs do match with the bores of the receptacle pipes!
3. Re-insert spring pins and ascertain that they have securely locked into place. Only if the pins stick out of the bores visibly and noticeably the shower chair has securely been height adjusted and an unwanted telescoping of the pipes can be prevented. (ill. 1)
4. Please ascertain that all seat legs have been adjusted at the same height. With the aid of the finely adjustable foot you can compensate for slight unevenness on the floor. To do this rotate the foot until both chair legs are in firm and safe contact with the floor.

Cleaning/Disinfection

The shower seat should only be used on intact skin. A cleaning agent that is gentler to material should be used for manual cleaning. Only DGHM-/VAH-listed material-friendly surface disinfectant should be used. Please see the relevant manufacturer's instructions for a selection of suitable cleaning agents and disinfectants and their use.

Re-use

The shower seat can be re-used. It must be cleaned and disinfected according to instructions. It must be inspected by specialist shops for any faults and then authorised before it can be used again. Please order our hygiene and maintenance guidelines.

Material

- Seat: PE
- Frame: powder coated aluminium tube
- Chair leg stoppers: rubber

Disposal

Please ask your specialist dealer for the regional disposal regulations.

Technical Details*

- Dimensions seat: 41 x 40 cm (L x D)
- Total width: 46 cm
- Total depth: 50 cm
- Seat height: 49-56,5 cm
- Number of height adjustments: 4
- Max. user body weight: 130 kg
- Weight: 2,2 kg

* Please allow for slight deviations.

This device complies with Regulation (EU) 2017/745 on medical devices and bears a CE marking.

If a serious incident occurs involving this medical device, contact the manufacturer of this device at info@russka.de as well as the competent authority, the Medicines & Healthcare products Regulatory Agency MHRA, www.gov.uk/government/organisations/medicines-and-healthcare-products-regulatory-agency.

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