REF ART.-NR. 07804
ACHILLO-HIT®

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INSTRUCTIONS FOR USE EN

Dear patient,

We ask you to carefully follow the enclosed instructions for
use. If you have any questions, please contact the doctor
looking after you, your nearest specialist store or us direct-
ly.

INTENDED USE

ACHILLO-HIT® is a support for Achilles tendon compres-
sion with large SPORLASTIC AIR-MATRIX silicone pads,
which have a massaging effect on the Achilles tendon and
relieve pressure.

INDICATIONS

- Conservative/postoperative
- Achillodynia with/without partial rupture
- Peritendinosis, peritendinitis (chronic/acute)
- Bursitis subachillea
- Bursitis calcanea

CONTRA-INDICATIONS

- Severe arterial circulatory disorders.
- Allergic, inflammatory or injury-related skin changes
(e.g. swelling, redness) of the body areas to be supplied.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenic disorders of the sensory system and skin
trophism in the body area to be treated (sensory disor-

- ders with or without skin damage).
- Severe arterial circulatory disorders.

SIDE EFFECTS

If used appropriately and correctly, there are no general
side effects, intolerances with long-term effects or allergic
reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indi-
cations and contraindications. Instruction and allocation
of the correct size is carried out by medical professionals.

PUTTING IT ON AND USE

- ACHILLO-HIT® is available for the left and right foot.
- Open the Velcro fasteners to enter ACHILLO-HIT® 1.
- Grasp the side parts of the Velcro fastener with both
hands 2.
- First pull the support carefully over the back of the foot.
Then grasp the back part of the support and pull it over
the heel 3.
- The control strip must be positioned centrally on the back
of your foot. Also, the back seam of the support should
be centred over the heel bone towards the calf 4.
- The extended pressure pad of the pad, which lies on the
inside of the foot, must run below the inner ankle and into
the arch of the foot. The shorter, outer pressure pad also
runs below the outer ankle 5.
- Close the Velcro fasteners in the area of the front lower
leg 6.

⚠ IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of
any orthopaedic device in general, as well as the pre-
sence of any of the conditions below in particular, should
be discussed with the doctor who is in charge of your
treatment.
- Check the functioning of the product with your doctor or
specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and
function, it may not be worn in conjunction with oily/gre-
asy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present prop-
erly. Damage caused by improper closing of the Velcro fas-
teners does not constitute a reason for complaint.

- In the event of serious incidents when using this aid, im-
mediately contact your doctor, your health care supply
retailer or contact SPORLASTIC directly at
info@sporlastic.de to report the incident.

- Any improper modifications to the product and/or im-
proper use of the above-mentioned product exclude any
product liability on the part of the manufacturer.
- Product may contain metallic objects. We recommend
not putting them in contact with electromagnetic interfe-
rence.

- With products involving direct skin contact with an elas-
tomer (e.g. a silicone pad), blistering may develop on the
skin if the product is worn too tightly or if it is moved
over the skin with high amounts of friction. In this case,
we recommend stopping use of the product. Sweat can
develop underneath the silicone, which in rare cases can
trigger a reaction to your own sweat with corresponding
allergic skin changes. SPORLASTIC is not aware of any
allergic reactions being caused by the material. If you
have any problems, please talk to your doctor or the sup-
plier of the aid.
- Possible health-related mutual risks or other disadvan-
tages with certain treatments that may arise in associati-
on with the use of the product must be discussed with the
doctor in charge of treatment.

- Please return the packaging and the product to the lo-
cal recycling collection system. Please comply with
local regulations.

- SPORLASTIC GmbH is affiliated with a returns system
for packaging and therefore fully complies with the
packaging legislation in the Federal Republic of Germa-
ny.

CARE

We recommend washing ACHILLO-HIT® gently in luke-
warm water (30° C) with mild detergent and air drying. Do
not dry the product in the tumble dryer, on the heating or in
the microwave. Close the Velcro fasteners before washing.
They will remain functional longer and damage to other
garments can be avoided.

Any tension in the knitted fabric that decreases over time
is restored by washing. We therefore recommend washing
the support more often.

☂☀ TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and

is protected against moisture and sunlight. Store the pro-
duct at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to pro-
duct inspection as part of our quality management system.
If you still have any complaints about our product, we kind-
ly ask that you contact your specialist retailer.

MATERIAL COMPOSITION

67% polyamide, 33% spandex

☒ An online version of the instructions for use can be
found on our website at www.sporlastic.de.

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