SPORLASTIC

REF 07506
EPI-HIT® ORGANIC

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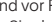
Contra-indications
All indications requiring relief of the tendon/muscle attachments at the ulnar and/or radial epicondyle, such as:

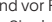
- Radial/ulnar epicondylitis (tennis elbow, golfer's elbow)
- Chronic or recurrent epicondylopathy
- Damage to the tendon/muscle attachments at the elbow with pain on exertion

Contra-indications
Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) in the areas of the body to be treated • Impaired circulation or lymphatic soft tissue swelling • Neurogenic disorders of sensory function and skin trophism in the area of the body to be treated (sensory disturbances with and without skin damage)

Side effects
If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

Intended patient target group
The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

Putting it on and use
• Open the Velcro fastener at the orange circular mark to its maximum width.
• Place your arm in front of you with your palm facing upwards. Pull the brace over your arm so that the fastening strap with the orange circular mark is positioned at the top of your forearm (tennis elbow , golfer's elbow )
• Position the pad, which can be felt under the padding, on the painful side of your forearm.
• Close the Velcro fastener with moderate tension (tennis elbow , golfer's elbow )
• The correct pressure is achieved when the brace is no longer noticeable after a few minutes with your arm relaxed and when pressure is felt under the pad when you tense your forearm muscles.

• The enclosed wrist support supports the effect of the brace. Open the Velcro fasteners and position the wrist band on your wrist. Close the Velcro fasteners with measured tension so that the wrist bandage fits snugly around your wrist (tennis elbow , golfer's elbow )
• To remove the brace, open the Velcro fastener at the orange mark and open the Velcro fasteners on the wrist support.
• EPI-HIT® ORGANIC should be worn during the day and removed at night.

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en Instructions for use

Dear patient,
We ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

Intended use
EPI-HIT® ORGANIC is an elbow brace designed to relieve pressure on the muscle origins at the ulnar and/or radial epicondyle.

Indications
All indications requiring relief of the tendon/muscle attachments at the ulnar and/or radial epicondyle, such as:

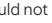
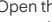
- Radial/ulnar epicondylitis (tennis elbow, golfer's elbow)
- Chronic or recurrent epicondylopathy
- Damage to the tendon/muscle attachments at the elbow with pain on exertion

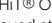
Contra-indications

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) in the areas of the body to be treated
- Impaired circulation or lymphatic soft tissue swelling
- Neurogenic disorders of sensory function and skin trophism in the area of the body to be treated (sensory disturbances with and without skin damage)

Side effects
If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

Intended patient target group
The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

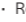
Putting it on and use
• Open the Velcro fastener at the orange circular mark to its maximum width.
• Place your arm in front of you with your palm facing upwards. Pull the brace over your arm so that the fastening strap with the orange circular mark is positioned at the top of your forearm (tennis elbow , golfer's elbow )
• Position the pad, which can be felt under the padding, on the painful side of your forearm.
• Close the Velcro fastener with moderate tension (tennis elbow , golfer's elbow )
• The correct pressure is achieved when the brace is no longer noticeable after a few minutes with your arm relaxed and when pressure is felt under the pad when you tense your forearm muscles.

• The enclosed wrist support supports the effect of the brace. Open the Velcro fasteners and position the wrist band on your wrist. Close the Velcro fasteners with measured tension so that the wrist bandage fits snugly around your wrist (tennis elbow , golfer's elbow )
• To remove the brace, open the Velcro fastener at the orange mark and open the Velcro fasteners on the wrist support.
• EPI-HIT® ORGANIC should be worn during the day and removed at night.

Transport and storage conditions 
Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

Quality management system
All products from SPORLASTIC GmbH & Co. KG are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

Materials used
Polyamide, elastane, polyurethane

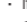
 An online version of the instructions for use can be found on our website at www.sporlastic.de.

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- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- If a significant deterioration in health occurs during the use of the medical device, we ask you to report this serious incident to your specialist dealer, your doctor or us as the manufacturer, as well as to the responsible authority. You can find our contact information in these instructions for use.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- With products involving direct skin contact with an elastomer (e.g. a silicone pad), blistering may develop on the skin if the product is worn too tightly or if it is moved over the skin with high amounts of friction. In this case, we recommend stopping use of the product. Sweat can develop underneath the silicone, which in rare cases can trigger a reaction to your own sweat with corresponding allergic skin changes. SPORLASTIC is not aware of any allergic reactions being caused by the material. If you have any problems, please talk to your doctor or the supplier of the aid.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

 Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

 SPORLASTIC GmbH & Co. KG is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

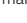
Care
We recommend washing EPI-HIT® ORGANIC gently in lukewarm water (30 °C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heating or in the microwave. The padding can be removed and cleaned. Close the Velcro fasteners when washing to ensure they remain functional for longer and to prevent damage to other items of clothing.

Transport and storage conditions 
Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

Quality management system
All products from SPORLASTIC GmbH & Co. KG are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

Materials used
Polyamide, elastane, polyurethane

 An online version of the instructions for use can be found on our website at www.sporlastic.de.

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