
EPIDYN[®] SUPREME

Ellenbogenbandage
Elbow support

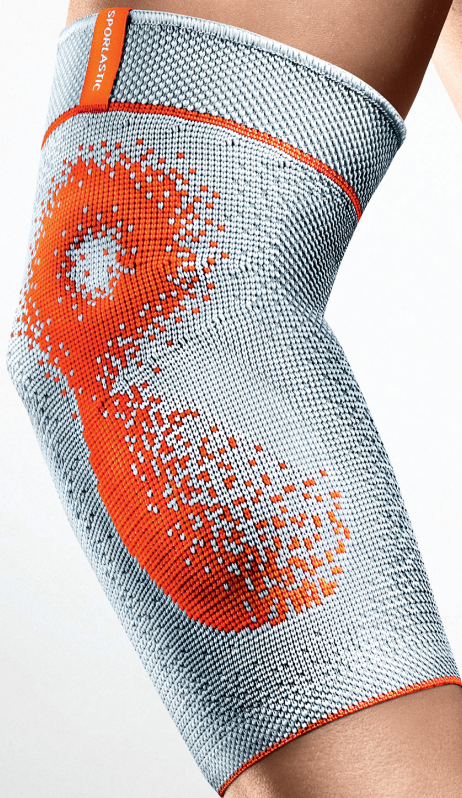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INSTRUCTIONS FOR USE **GB**

Dear Customer,

we would like to ask you to follow these Instructions for Use with care. If you should have any questions, please consult the doctor treating you or your nearest stockist, or contact us directly.

FUNCTION

EPIDYN® SUPREME relieves and compress the elbow joint. The integrated SPORLASTIC AIR-MATRIX 1 silicone friction pads with their nubbed surface towards the skin has a positive effect on painful insertion tendopathy, capsular irritation and epicondylopathy, which are common in arthrosis. The improved circulation speeds up the absorption of effusions.

INDICATIONS

- Epicondylitis
- Tendomyopathy
- Arthrosis of the elbow joint
- Ligament insufficiency
- Concomitant epicondylopathy with arthrosis
- Elbow distortion
- Irritations following operations, fractures, rheumatic elbow effusions

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far not consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

With proper use and proper fitting, so far there have been no reports of serious general side-effects.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restricting consistently shaped body fitting.

FITTING

- EPIDYN® SUPREME can be worn on either the left or right elbow.
- Position the ring-shaped cut-outs of the SPORLASTIC AIR-MATRIX silicone pads insert so that they rest against both sides of the exposed joint (lateral and medial epicondyle).
- Adjust the band pressure depending on your needs.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from productliability. Please make sure to always close the Velcro fasteners properly. As a matter of principle, damage arising from improper fastening of the Velcro fasteners is not accepted as grounds for claims.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

CARE

We recommend that EPIDYN® SUPREME be washed gently in lukewarm water with detergent for delicate items and that it then be air dried (do not dry on a heater).

TIP: Slackening tension of the knitted fabric while it is being worn is revived by washing. The bandage should therefore be washed frequently.

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