FIBULO-TAPE®

REF ART.-NR. 07073
FIBULO-TAPE®

Sprunggelenkbandage

Ankle support

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INSTRUCTIONS FOR USE EN

Dear Patient,

We ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

INTENDED USE

FIBULO-TAPE® is an ankle soft tissue compression support with local friction pads, additional 8-strap and inelastic lateral rein.

INDICATIONS

- Primary and secondary prevention of ankle trauma, especially in sports
- Therapy of acute ligament sprains and chronically

CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage).

SIDE EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

PUTTING IT ON AND USE

- First pull FIBULO-TAPE® over the foot. Make sure that the heel sits correctly in the support. The incorporated nubby silicone pads should surround the ankles at the back and bottom.
- For the function of FIBULO-TAPE®, stabilisation of the ankle joint, the correct application of the reins 1 to 3 is of great importance (Fig. 1-5).
- Place the reins numbered 1 to 3 next to your foot. Reins 1 and 3 on the outside, rein 2 on the inside of the foot (Fig. 1).
- Place the foot firmly on the ground. Push the knee or lower leg forward. Foot and lower leg should form an angle of 70-80 degrees.
- Now guide the partially elastic rein 1 diagonally over the instep and above the inner ankle around the ankle joint (fig. 2) and fix it under tension at the back on top of the small Velcro strap. The foot remains completely on the ground with the heel and the lower leg advanced.
- Now lead rein 2 in the opposite direction over the instep and then around the leg (fig. 3). Also under tension, fasten it above the inner ankle on the Velcro strap of rein 1 (8 strap guide).
- Lastly, guide the inelastic rein 3 upwards on the outside of the ankle with the traction. Lift the outer edge of the foot slightly. Then attach rein 3 to the Velcro strap of rein 2. Rein 3 should run vertically or slightly diagonally upwards to the rear (Fig. 4).
- To achieve the correct hold and direction of pull, tighten the reins again if necessary. Make sure that there are no creases on the reins or that the reins are tight and without creases on the foot and leg.
- In combination with a ready-made (lace-up/sport) shoe, the stability function of FIBULO-TAPE® receives additional reinforcement (Fig. 6).

NOTES FOR THE ORTHOPAEDIC TECHNICIAN

- If the reins are optimally applied (Fig. 5), any protruding lengths can be shortened.

IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- With products involving direct skin contact with an elastomer (e.g. a silicone pad), blistering may develop on the skin if the product is worn too tightly or if it is moved over the skin with high amounts of friction. In this case, we recommend stopping use of the product. Sweat can develop underneath the silicone, which in rare cases can trigger a reaction to your own sweat with corresponding allergic skin changes. SPORLASTIC is not aware of any allergic reactions being caused by the material. If you have any problems, please talk to your doctor or the supplier of the aid.
- Possible health-related mutual risks or other disadvantages

tages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

CARE

We recommend washing FIBULO-TAPE® gently in lukewarm water (30°C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heating or in the microwave. To avoid damaging other garments during washing, it is best to wash FIBULO-TAPE® separately or close the Velcro fasteners beforehand.

Any tension in the knitted fabric that decreases over time is restored by washing. We therefore recommend washing the support more frequently.

TRANSPORT AND STORAGE CONDITIONS
Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

MATERIAL COMPOSITION

67% Polyamid, 33% Elasthan

An online version of the instructions for use can be found on our website at www.sporlastic.de.

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