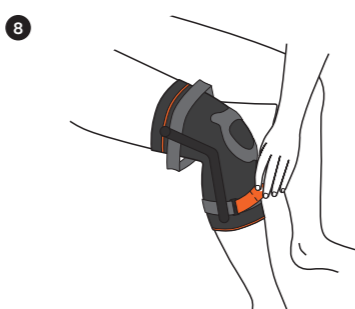
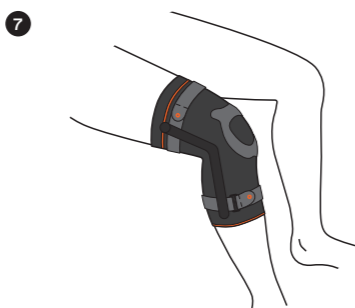
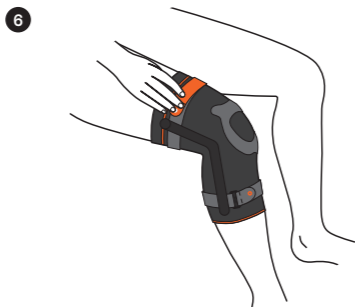
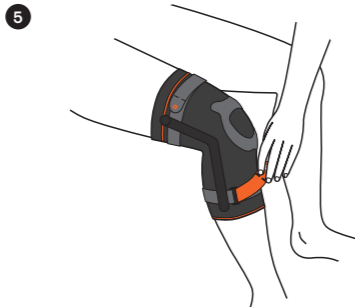
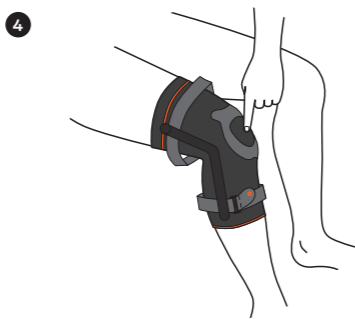
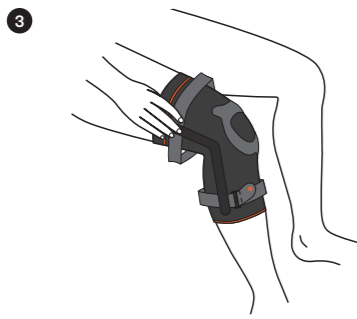
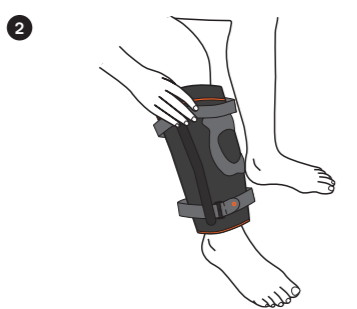
**07105**  
GENU-HIT® GS

- de** Knieorthese mit seitlichen Gelenkschienen, extrawide Version, zusätzlich mit zirkulärem Silikonknochenhaftband
- en** Knee brace with lateral joint splints, extra wide version, additionally with circular silicone nub adhesive tape
- fr** Orthèse de genou avec attelles latérales, version extra-large, avec bande adhésive circulaire en silicone supplémentaire
- it** Ortesi per ginocchio con stecche laterali, versione extra larga, con nastro adesivo circolare in silicone
- pl** Orteza kolana z bocznymi szynami stawowymi, wersja extra szeroka, dodatkowo z okrągłą silikonową taśmą antypoślizgową
- da** Kneørtose med laterale ledskinner, ekstra bred version, desuden med cirkulært silikonebånd med nopper
- sl** Kolenska ortoza z bočnimi opornimi letvami, ekstra široka različica, dodatno opremljena s krožnim silikonskim trakom za oprjem
- et** Põlveortoos külgmise liigese kinnitusega, eriti lai versioon, lisaks ringikujulise silikoonist kinnitusbaga

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## en Instructions for use

Dear patient, We ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

**Intended use**  
GENU-HIT® GS/GENU-HIT® GS + COMFORT is a knee brace for physiological guidance of the knee joint and relief with protection of the joint apparatus.

**Indications**  
All indications where support of the physiological guidance of the knee joint is necessary, such as:  

- Conservative/postoperative
- Mild to moderate lateral ligament instability of the knee joint
- Damage to capsule/ligament structures of the knee joint
- Combined ligament injuries of the knee
- Mild ligament insufficiencies, strains, ruptures
- Mild instabilities
- Collateral ligament lesions
- After cruciate ligament injury
- Osteoarthritis of the knee
- Arthritis
- Irritation
- Baker's cysts

**Contra-indications**  
In general, the treating physician should be consulted regarding the indication and wearing mode of an orthopaedic aid, both in general and in the presence of the following medical conditions in particular:  

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) in the areas of the body to be treated
- Impaired circulation or lymphatic soft tissue swelling
- Neurogenic sensory and skin trophic disorders in the body area to be treated (sensory disorders with and without skin damage)

**Side effects**  
If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

**Intended patient target group**  
The target group is all patients, taking into account indications and contraindications: Instruction and allocation of the correct size is carried out by medical professionals.

**Putting it on and use**  

- Open the Velcro fasteners and loosely attach the straps to the suede ❶ to prevent damage to the knit.
- Hold the knee brace at the top on both sides ❷ and pull the GENU-HIT® GS/GENU-HIT® GS + COMFORT over your knee ❸.
- Make sure that your kneecap is not pressed against the SPORLASTIC AIR-MATRIX silicone friction pad ❹. The two knee braces should not exert any uncomfortable pressure on the leg.
- First close the lower strap with moderate tension ❺. Repeat this with the upper strap ❻.
- Ensure that the edge of the brace is not folded inwards or outwards and lies flush against the body ❼.
- To remove the brace, open the straps and loosely fasten the straps to the leg ❶.
- Now slide the GENU-HIT® GS/GENU-HIT® GS + COMFORT down from the top towards the foot ❷.

**Verwendetes Material**  
Polyamide, silicone, elastane, polyurethane

**Online version**  
An online version of the instructions for use can be found on our website at [www.sporlastic.de](http://www.sporlastic.de).

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**Important information**  

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.

- The product can come into contact with splash water.
- Gonarthrose
- Arthrites
- États irritatifs
- Kystes de Baker

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