SPORLASTIC®

SPORLASTIC®

ART.-NR. 07081
GENU-HIT®

ART.-NR. 07087
GENU-HIT®+ COMFORT

ART.-NR. 87081
GENU-HIT® KIDS

ART.-NR. 07083
GENU-HIT®+

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67% Polyamid, 33% Lycra

INSTRUCTIONS FOR USE GB

Dear Customer,
we would like to ask you to follow these Instructions for Use with care. If you should have any questions, please consult the doctor treating you or your nearest stockist, or contact us directly.

FUNCTION

GENU-HIT®, GENU-HIT®+, GENU-HIT®+ COMFORT, GENU-HIT®RS and GENU-HIT® WING are compression bandages with lateral spirals and a local silicon profile padding around the kneecap.

GENU-HIT®, GENU-HIT®+, GENU-HIT®+ COMFORT, GENU-HIT®RS and GENU-HIT® WING support proprioception and lead to the improved muscular stability of the knee. The ring shaped silicon profile padding around the kneecap supports the resorption of oedema and (rheumatic) effusions.

GENU-HIT®+ expands the spectrum of knee care with four additional interim sizes for the optimal treatment of overproportional circumferences in the thigh. The knit fabric adjusts anatomically to the contours of the thigh area, thus providing a good fit and making it quite comfortable to wear.

GENU-HIT®+ COMFORT: The additional silicone adhesion strip around the top of the bandage provides a lasting hold and perfect positioning around the knee.

GENU-HIT®RS: Additionally integrated burlled silicone adhesion strips ensure a permanent, accurately fitting hold around the knee.

INDICATIONS

- Patellar chondropathy
- Meniscus injuries
- Gonarthrosis
- Postoperative and post-traumatic irritation

- Rheumatic effusions with and without ligamentary insufficiency
- Ligament injuries
- Slight ligamentary insufficiency

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far no consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

To date, there have been no reports of serious general side effects with proper use and fitting.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restrictive, consistently shaped shaped body fitting.

FITTING

Grasp the knee bandage at both upper ends of the lateral spirals and pull them over the knee so that the circular silicone padding envelops the kneecap without applying pressure on it.

INNOVATION MAKES IT COMFORTABLE TO WEAR

GENU-HIT®, GENU-HIT®+, GENU-HIT®+ COMFORT, GENU-HIT®RS and GENU-HIT® WING are very comfortable to wear with an ideal positioning almost entirely free of creases thanks to the accordion effect in the hollow of the knee.

The essence of the innovation is an oval intarsia element **1**. When the knee is stretched the intarsia element extends, when the knee flexes the stitching contracts - without any gathering of material in the popliteal cavity. This has noticeable medical benefits for the patient.

The knee bandages GENU-HIT®, GENU-HIT®+, GENU-HIT®+ COMFORT, GENU-HIT®RS and GENU-HIT® WING are made from latex-free material. The use of Lycra provides continuous compression.

Microfiber-like material makes the bandages very comfortable to wear. The material used in the knit fabric provides the basis for a very pleasant micro climate which keeps the bandage comfortable to wear even during times of physical activity.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

GENERAL NOTE

The use of products involving direct exposure of the skin to an elastomer (e.g. a silicone pad) might possibly lead to blisters on the skin if the product is applied too tightly or if it is shifted on the skin at high friction. In this case, we recommend to discontinue the use of the product. Sweat may be generated under the silicone and, in rare cases, this can lead to the body reacting to its own sweat and cause corresponding allergic skin changes. Sporlastic is not aware of any allergic reactions related to the materials. If you experience any problems, please contact your doctor or the supplier of the medical aid.

CARE

Wash GENU-HIT®, GENU-HIT®+, GENU-HIT®+ COMFORT, GENU-HIT®RS and GENU-HIT® WING in lukewarm water with a coldwater detergent and air dry. Do not place on a radiator.

TIP: Slackening tension of the knitted fabric while it is being worn is revived by washing. The bandage should therefore be washed frequently.

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