MALLEO-CAST® ORGANIC S1

SPORLATIC

07849

MALLEO-CAST® ORGANIC S1

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Verwendetes Material

Polyamid (PA), Polyester (PES), Elasthan (EL)

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en Instructions for use

Dear patient,

We ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor locking after you, your nearest specialist store or us directly.

Intended use

MALLEO-CAST® ORGANIC S1 is a brace for immobilising and stabilising the upper and/or lower ankle joint in a defined position.

Indications

Damage to bone/capsule/ligament structures of the ankle joint (as a result of injuries such as ligament rupture, severe ankle sprain, after surgery)

- For temporary immobilisation and stabilisation
- All indications requiring immobilisation of the ankle joint, such as:
 - Supination/pronation trauma
 - Postoperative protection after ligament suturing/reconstruction
 - Conservative treatment of severe ankle sprains
 - Complex instability of the upper/lower ankle joint
 - Acute/ligament injuries to the ankle joint
 - Immobilisation of the upper/lower ankle joint
 - Relief/protection of injured capsule/ligament structures
 - Follow-up care for fractures in the ankle joint area after immobilisation with a lower leg/foot brace
 - Immobilisation in cases of inflammatory irritation

Contra-indications

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) in the areas of the body to be treated
- Impaired circulation or lymphatic soft tissue swelling
- Neurogenic disorders of sensory function and skin trophism in the area of the body to be treated (sensory disturbances with and without skin damage)
- Peripheral AVK
- Borderline peripheral blood circulation reserve
- Risk of vascular compression

Side effects

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

Intended patient target group

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

Putting it on and use

- We recommend wearing MALLEO-CAST® ORGANIC S1 over a long sock.
- Open the magnetic twist lock by lifting the wheel and open the rear upper Velcro fastener. To prevent the free end of the Velcro from sticking to free fleece surfaces, stick it onto the same strap before.
- All front fasteners are for adjustment only and do not need to be opened for putting on and taking off.

- Step through the front pad opening of the MALLEO-CAST® ORGANIC S1 with the tip of your foot. Avoid lowering the tip of your foot to protect your injured structures.
- Pull the brace as far as possible onto your foot until the in-strap (back of the foot) touches the front straps. Move your foot back so that your ankle joint is at a 90° angle.
- Put the dial back on and tighten the rear fastener by turning it clockwise. Then close the rear upper Velcro fastener. The Velcro end should land on the front upper hook-and-loop surface.
- If you find the fit uncomfortable, optimise the position of the brace by tightening or loosening the front fasteners and retightening the rotary wheel fastener.
- To remove the brace, simply open the rotary wheel fastener and the rear upper Velcro fastener. Then pull the brace forward off your foot.

Notes for the orthopaedic technician

- When correctly fitted, the outer ankle should be positioned in the centre of the lateral shell opening.
- When delivered, all front fasteners are closed in the widest position. Once the patient has pulled the brace over their foot, ensure that the ankle joint is in a 90° position.
- Close the rear fastener using the rotary wheel and tighten it until there is a reserve of 1 – 1.5 cm.
- Then adjust the tension of the front upper fastener.
- Next, grasp the two Velcro fasteners of the 8-point strap that is set too wide and tighten both ends simultaneously without pulling the stabilising shells forward.
- Finally, close the upper rear Velcro fastener strap.
- Check the position of the outer ankle in the recess.
- If necessary, correct the position of the outer shell by loosening or retightening the front fasteners and the rotary wheel fastener.
- If there is a specific pressure point, the brace shell can be heated in a hot water bath at approx. 90 °C and then reshaped. Please remove the padding beforehand.

Spare parts

Optionally available from your specialist retailer:

- Replacement cushion for MALLEO-CAST® ORGANIC S1 (07849)

Important information

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- Please do not use disinfectants.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closingof the Velcro fasteners does not constitute a reason for complaint.
- If a significant deterioration in health occurs during the use of the medical device, we ask you to report this serious incident to your specialist dealer, your doctor or us as the manufacturer, as well as to the responsible authority. You can find our contact information in these instructions for use.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- Product may contain metallic objects. We recommend not putting them in contact with electromagnetic interference.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

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Please return the packaging and the product to the local recycling collection system. Please comply with local regulations

SPORLATIC GmbH & Co. KG is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

Care

We recommend washing MALLEO-CAST® ORGANIC S1 gently in lukewarm water (30 °C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heating or in the microwave. The padding can be removed from the brace at the orange plastic clips for cleaning.

Transport and storage conditions

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

Quality management system

All products from SPORLATIC GmbH & Co. KG are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

Materials used

Polyamide (PA), polyester (PES), elastane (EL)

An online version of the instructions for use can be found on our website at www.sporlastic.de.

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