MANU-HIT®

SPORLASTIC®

ART.-NR. 07035

MANU-HIT®

ART.-NR. 87035

MANU-HIT® KIDS

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INSTRUCTIONS FOR USE GB

Dear Customer,

we would like to ask you to follow these Instructions for Use with care. If you should have any questions, please consult the doctor treating you or your nearest stockist, or contact us directly.

FUNCTION

MANU-HIT® has a stabilising, movement restricting function for the wrist joint. This is achieved by volar (on the inside of the hand) and dorsal (on the back of the hand) stabilizing splints. In this way the loads on the wrist bones, on the articular capsules and on the ligaments is reduced.

The wide volar splint is anatomically preformed to stabilize the hand and can be adapted individually as required.

INDICATIONS

- Rheumatoid arthritis
- Tendovaginitis
- Carpal-canal-syndrome
- After removal of plaster casts
- After severe sprains
- Overstrain arthropathy with or without incongruence
- Arthrosis of the wrist
- Lunatomalacia
- Navicular pseudarthrosis
- Drop-wrist

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far no consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

To date, there have been no reports of serious general side effects with proper use and fitting.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restrictive, consistently shaped shaped body fitting.

FITTING

- Slide your hand into the brace **1**.
- Close the Velcro fasteners **2**.
- Now guide the Velcro tape **3** between your thumb and forefinger and fix this.
- The inelastic wide Velcro tape **4** reinforces the stabilization of the wrist by dorsal circular tension.
- Finally put on the Velcro tapes **2** so that the brace sits firmly against your hand without pressing unpleasantly. If necessary correct the Velcro tape **3** as well.

COMFORTABLE TO WEAR

MANU-HIT® is made of the most modern materials and combine the ultimate in design with perfect function and wearing comfort.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in con junction with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

CARE

Wash MANU-HIT® in lukewarm water with a cold-water detergent and air dry. Do not place on a radiator. The enclosed heel pad can be cleaned with a moist cloth. When washing please remove the two splints from the brace. Close the Velcro fasteners before washing. The Velcro fasteners will then last longer.

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