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INSTRUCTIONS FOR USE GB

Dear Customer,

We would like to ask you to follow these Instructions for Use with care. If you should have any questions, please consult the doctor treating you, at your nearest stockist or contact us directly.

FUNCTION

OMO-LUX® is a shoulder brace for the conservative or postoperative treatment of the shoulder. It is a stabilizing and restricting orthotic device utilizing special harness guides.

OMO-LUX® allows free movement in the ante version, limited movement in the retroversion due to the safety harness and more than 90° in abduction due to the torsion joint.

INDICATIONS

- _ Habitual shoulder dislocation, I and II degree injuries of the shoulder joint after Tossi
- _ Postoperative treatment of ruptured ligaments after Tossi III
- _ Postoperative treatment after distal claviclectomy
- _ Irritations caused by arthrosis

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- _ Do not use as direct injury treatment after Tossi III.
- _ Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- _ Impaired circulation or lymphatic soft tissue swelling.
- _ Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far no consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

To date, there have been no reports of serious general side effects with proper use and fitting.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restrictive, consistently shaped body fitting.

FITTING

- _ Open the underarm, upper arm and chest straps (front) of the OMO-LUX® and fix the ends to the fleece.
- _ Attach the OMO-LUX® (consisting of two plastic shells) to the affected shoulder. Ensure that the brace has a tight fit.

_ First, secure the chest and then the upper arm strap fairly lightly.

_ Then close the axillary strap as tightly as can be tolerated.

_ Then retighten the chest and upper arm straps.

TIP: The connecting strap between the two shells can be individually adjusted as needed. The shorter the closure is adjusted, the less one is able to achieve the retroversion, i.e. the arm movement backwards.

NOTE FOR THE ORTHOPAEDIC TECHNICIAN

The fit can be adjusted, if necessary, by cutting or grinding the two shells to the individual fit. The material can also be thermoformed at 140° C.

The padding provided should be left as far as possible, as the existing overhang helps to reduce pressure at the edges.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

CARE

We recommend cleaning the plastic parts of the OMO-LUX® with a damp cloth and to wash the padded parts carefully in lukewarm water with a mild detergent, allow to dry in the air and not to put on the radiator. Close the Velcro closures before washing. These remain functional for a longer period and damage to other pieces of clothing can be avoided.

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SPORLASTIC GmbH Medizinische Produkte