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INSTRUCTIONS FOR USE EN

Dear patient, we ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

INTENDED USE

RHIZO-RING® / RHIZO-RING® SOFT is a brace for stabilising and / or immobilising the metacarpophalangeal joint of the thumb and thumb saddle joint and is used for rhizarthrosis, injuries and ligament instabilities.

INDICATIONS

[REF] 07601, RHIZO-RING®:

- Conservative / postoperative
- Post-traumatic
- Degenerative inflammation
- Early-onset to slightly advanced rhizarthrosis with tendency towards subluxation
- Functional symptoms in the carpo-metacarpal joint caused by recurrent joint misalignment

[REF] 07603, RHIZO-RING® SOFT:

- Conservative / postoperative

- Post-traumatic
- Degenerative inflammation
- Early-onset to slightly advanced rhizarthrosis with tendency towards subluxation
- Functional symptoms in the carpo-metacarpal joint caused by recurrent joint misalignment
- Inflammation of the carpo-metacarpal joint

CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied
- Impaired circulation or lymphatic soft tissue swelling
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage)

SIDE EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

PUTTING IT ON AND USE

- The product is delivered with two fastening straps for changing.
- To put on the brace, spread the thumb slightly.
- Insert the thumb through the opening of the ring. Make sure that the ball of the thumb rests on the ergonomically shaped wide contact surface of the ring.
- RHIZO-RING® / RHIZO-RING® SOFT should be placed positively between the thumb and the index finger.
- Guide the strap through the integrated guide slot and position the brace at the thumb saddle joint by applying a measured amount of tension to the strap over the back of the hand and then tying it.
- Apply as much tension to the strap as necessary until you achieve pain reduction and repositioning of the thumb saddle joint via the RHIZO-RING® / RHIZO-RING® SOFT.
- The touch of the fingertips with the thumb is supported (opposition grip), grasping becomes possible again without pain.

NOTES FOR THE ORTHOPAEDIC TECHNICIAN

- For an optimal fit, the thumb splint can be reworked by the orthopaedic technician (grinding with a funnel cutter or hand sandpaper). Thermal deformation is only possible to a limited extent.

ACCESSORIES / SPARE PARTS

Optionally available from your specialist store:

- Replacement strap for RHIZO-RING® and RHIZO-RING® SOFT ([REF] 17601)

▲ IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- If a significant deterioration in health occurs during the use of the medical device, we ask you to report this serious incident to your specialist dealer, your doctor or us as the manufacturer, as well as to the responsible authority. You can find our contact information in these instructions for use.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

☒ An online version of the instructions for use can be found on our website at www.sporlastic.de.

CARE

We recommend cleaning RHIZO-RING® / RHIZO-RING® SOFT with a damp cloth and washing the drawstrings gently in lukewarm water (30°C) with mild detergent and air drying. Do not dry the product in a tumble dryer, on the heating or in the microwave.

☂☀ TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and humidity. Store the product at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

☒ An online version of the instructions for use can be found on our website at www.sporlastic.de.

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