SP AIR WALKER PLUS

SPORLASTIC







07973

SP AIR WALKER PLUS

de	Unterschenkel-Fuß-Orthese zur Immobilisierung in definierter Spitzfußstellung
en	Lower leg and foot brace for immobilisation in a defined pointed foot position
fr	Orthèse de jambe et de pied pour l'immobilisation dans une position définie du pied en pointe
it	Ortesi per gamba e piede per l'immobilizzazione in una posizione definita del piede cavo
pl	Orteza podudzia i stopy do unieruchomienia w określonej pozycji stopy końsko-szpotawej
da	Underben-foerd-ortose til immobilisering i defineret spidsfoedstilling
sl	Ortéza za spodnji del noge in stopala za imobilizacijo v določeni položaju stopala

Anlegungsanleitung
Fitting instructions
Produktinfos
Product information

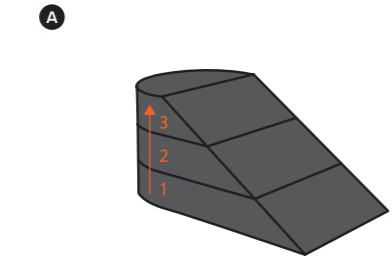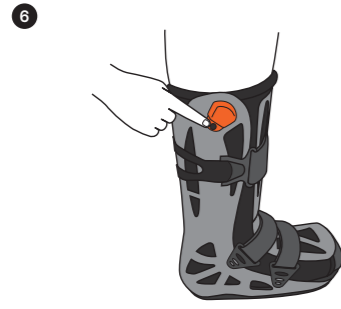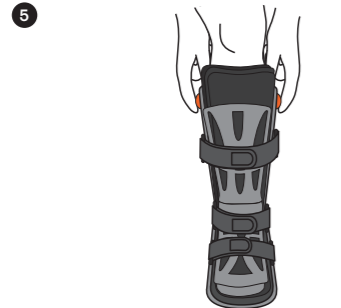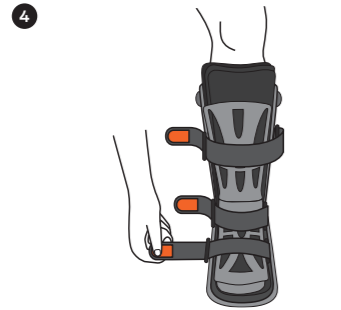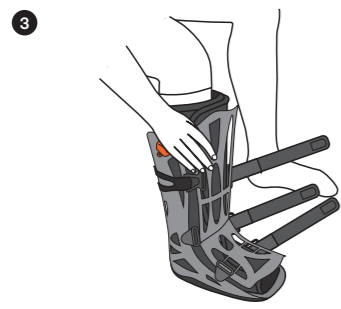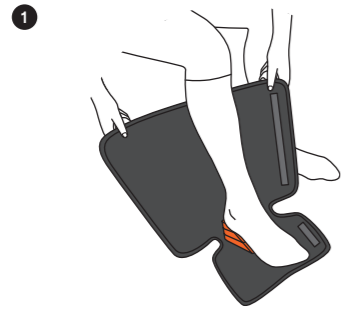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en Instructions for use

Dear patient,

We ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

Intended use

SP AIR WALKER PLUS is a lower leg foot brace for immobilisation in defined, adjustable positions.

Indications

- All indications requiring immobilisation of the lower leg, ankle joint and all or part of the foot in defined, adjustable positions in cases of damage to bone, capsule or ligament structures of the ankle joint and foot, such as:
 - Preoperative
 - Postoperative
 - Post-traumatic
 - Achilles tendon lesion
 - Achilles tendon rupture
 - Tendon injuries
 - Stable ankle fractures
 - Prosthetics
 - Arthroses
 - Severe ankle sprains
 - Severe ligament injuries

Contra-indications

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) in the areas of the body to be treated

- Impaired circulation or lymphatic soft tissue swelling
- Neurogenic disorders of sensory function and skin trophism in the area of the body to be treated (sensory disturbances with and without skin damage)
- Unstable fractures or fractures of the proximal tibia or fibula
- Existing lymphatic drainage disorders

Side effects

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

Intended patient target group

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

Putting it on and use

- We recommend wearing the brace with an undergarment.
- Open the Velcro fasteners, remove the front plastic cover and open the Velcro fasteners on the pad.
- Check that the wedge set is positioned at the very back of the pad or plastic shell.
- Schließen Sie dann das Polster im Bereich des Unterschenkels. Das Polster sollte das Bein faltenfrei und ganzflächig umschließen.
- Place your foot into the SP AIR WALKER PLUS from the front.
- Ensure that your foot and lower leg are free of pressure and lie completely at the back of the shell or pad. The heel rests on the inner heel wedges, which are located at the very back of the pad.
- Position the front plastic cover in the centre of the pad. Now close all Velcro fasteners over the plastic cover. Start with the foot strap.
- To immobilise your foot securely, the straps should be tightened firmly.
- The integrated air chambers can be inflated separately using the pump system and support the guidance and positioning of the foot in the walker.
- To inflate, apply repeated pressure to the respective orange bellows pump.
- To release the air, press the black vent valve at the bottom of the orange bellows.
- Finally, check once again that the brace fits without applying pressure.
- If necessary, the toe protection on the front plastic cover can be moved or removed.
- To remove the brace, follow the steps in reverse order.
- During the course of therapy, the heel wedges are removed in consultation with the attending physician.
- With maximum pes cavus, it is almost impossible to walk safely with the walker. During this early phase, the brace should be understood as a positioning shell.
- After reducing the brace to a less pronounced pes cavus position, it may gradually become possible to walk with partial weight-bearing, in consultation with the attending physician. Under certain circumstances, it may be advisable to compensate for leg length on the opposite side.
- Be particularly careful of the risk of slipping on wet surfaces!

- any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- If a significant deterioration in health occurs during the use of the medical device, we ask you to report this serious incident to your specialist dealer, your doctor or us as the manufacturer, as well as to the responsible authority. You can find our contact information in these instructions for use.

- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- With products involving direct skin contact with an elastomer (e.g. a silicone pad), blistering may develop on the skin if the product is worn too tightly or if it is moved over the skin with high amounts of friction. In this case, we recommend stopping use of the product. Sweat can develop underneath the silicone, which in rare cases can trigger a reaction. SPORLASTIC is not aware of any allergic reactions being caused by the material. If you have any problems, please talk to your doctor or the supplier of the aid.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

SPORLASTIC GmbH & Co. KG is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

Care

We recommend washing SP AIR WALKER PLUS gently in lukewarm water (30 °C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heating or in the microwave. The padding can be completely removed from the brace for cleaning. Please clean the plastic parts with a damp cloth.

Transport and storage conditions

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

Quality management system

All products from SPORLASTIC GmbH & Co. KG are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

Materials used

Polyethylene (PE), polyurethane (PU), polyamide (PA), thermoplastic polyurethane (TPU), acrylonitrile butadiene styrene (ABS), polyacetals (POM), polypropylene (PP), thermoplastic elastomer (TPE), ethylene vinyl acetate (EVA), spandex

An online version of the instructions for use can be found on our website at www.sporlastic.de.

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