
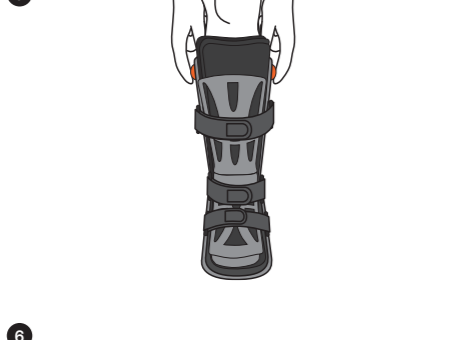
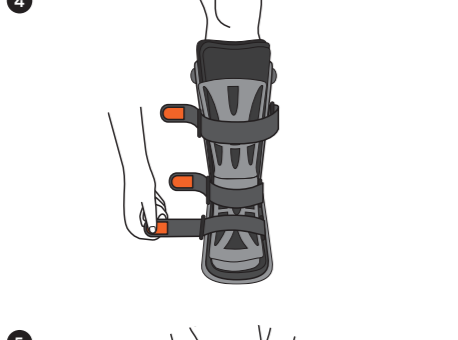
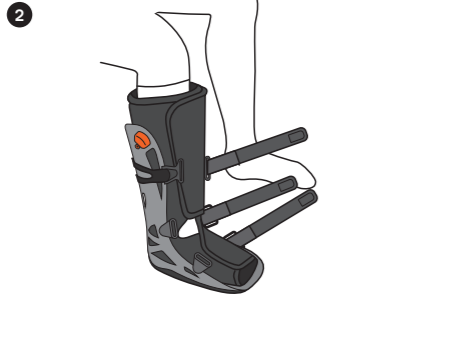
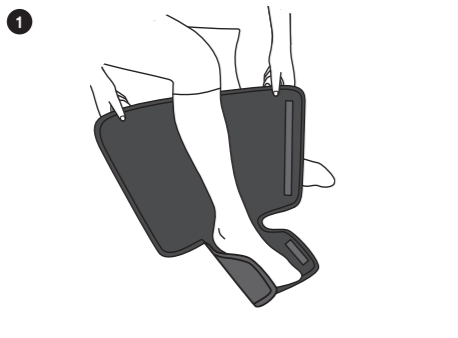
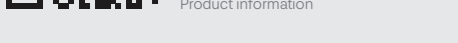
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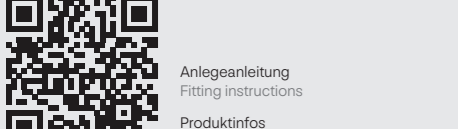

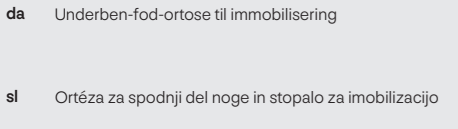
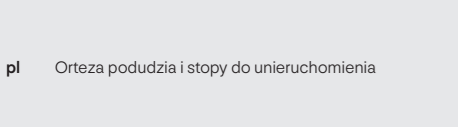
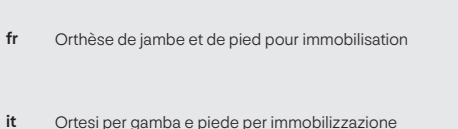
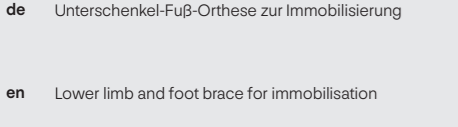
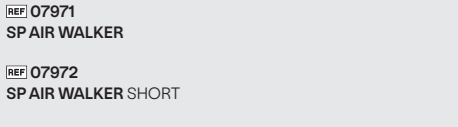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
## SP AIR WALKER

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☛ 07972, SP AIR WALKER SHORT: Alle Indikationen involving damage to bone/capsule/ligament structures of the foot or ankle joint as a result of injury or following surgery, such as:

- Distal fibula fractures
- Ankle fractures
- Tarsal and metatarsal fractures
- Stress fractures
- Severe ankle sprains
- Severe ligament injuries
- Ligament injuries
- Soft tissue injuries
- Tendon injuries
- Stable foot and ankle fractures

☞ 07972, SP AIR WALKER SHORT: All indications involving damage to bone/capsule/ligament structures of the foot or ankle as a result of injury or following surgery, such as:

- Conservative/postoperative
- Ligament injuries
- Soft tissue injuries
- Tendon injuries
- Stable foot and ankle fractures

**Contra-indications**

- Allergic, inflammatory, or injury-related skin changes (e.g., swelling, redness) in the areas of the body to be treated
- Impaired circulation or lymphatic soft tissue swelling
- Neurogenic disorders of sensory function and skin trophism in the area of the body to be treated (sensory disturbances with and without skin damage

**Side effects**
If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

**Intended patient target group**

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

**Putting it on and use**

- We recommend wearing the brace with an undergarment.

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**Notes for the orthopaedic technician**

- During initial fitting, the Velcro fasteners can be adjusted to the desired length or trimmed at the end where they are threaded through.
- The toe pad is attached to the underside of the padded inlay and can be moved or removed depending on the indication or season.
- The width of the lower leg shaft can be individually adjusted. To do this, unfold the corresponding strap deflection loops and then adjust the black plastic tension strap on the rubbed field. The new position is secured by folding the deflection loops back in.
- The toe protection on the front plastic cover can be adjusted to the respective foot length.

**Spare parts**

Optionally available from your specialist retailer:

- Replacement cushion SP AIR WALKER ☞ 17971
- Replacement cushion SP AIR WALKER SHORT ☞ 17972

**Accessories**

Optionally available from your specialist retailer:

- Rain protection ☞ 67966
- Hygiene protection ☞ 37966
- Under-knee stocking ☞ 87966

**Important information** ☛

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closingof the Velcro fasteners does not constitute a reason for complaint.
- If a significant deterioration in health occurs during the use of the medical device, we ask you to report this serious incident to your specialist dealer, your doctor or us as the manufacturer, as well as to the responsible authority. You can find our contact information in these instructions for use.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- With products involving direct skin contact with an elastomer (e.g. a silicone pad), blistering may develop on the skin if the product is worn too tightly or if it is moved over the skin with high amounts of friction. In this case, we recommend stopping use of the product. Sweat can develop underneath the silicone, which in rare cases can trigger a reaction to your own sweat with corresponding allergic skin changes. SPORLASTIC is not aware of any allergic reactions being caused by the material. If you have any problems, please talk to your doctor or the supplier of the aid.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

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☛ Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

☛ SPORLASTIC GmbH & Co. KG is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

**Care**
We recommend gently washing the cushion of SP AIR ALKER/SP AIR WALKER SHORTin lukewarm water (30 °C) with mild detergent and allowing it to air dry. Do not dry the product in a tumble dryer, on a radiator, or in a microwave. The padding can be completely removed from the brace for cleaning. Please clean the plastic parts with a damp cloth.

**Transport and storage conditions** ☛☛

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

**Quality management system**

All products from SPORLASTIC GmbH & Co. KG are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

**Materials used**

Polyethylene, polyurethane, polyamide, thermoplastic polyurethane, acrylonitrile butadiene styrene, polyacetals, polypropylene, thermoplastic elastomer, ethylene vinyl acetate, spandex

☛ An online version of the instructions for use can be found on our website at [www.sporlastic.de](http://www.sporlastic.de).

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