
## SP POSTURE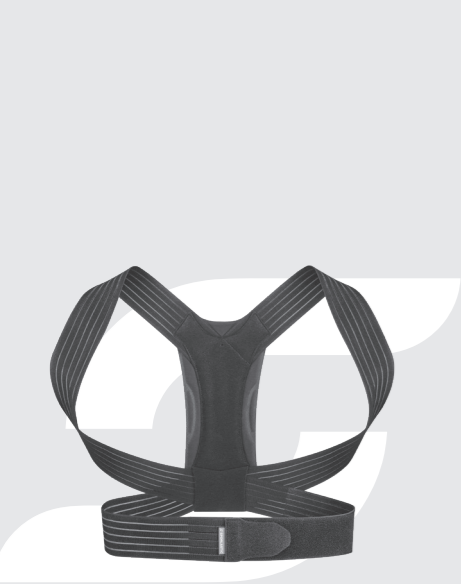

**07136**  
**SP POSTURE**

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**Verwendetes Material**

Polyamid, Polyurethan, Polyester, Elasthan

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# en Instructions for use

Dear patient,

We ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

**Intended use**

SP POSTURE is a thoracic spine brace designed to straighten and relieve the thoracic spine.

**Indications**

- Damage to the thoracic spine
  - For straightening the thoracic spine
- All indications where straightening and relief of the thoracic spine is necessary, such as:
- Poor posture of the thoracic spine
  - Postural weakness
  - Dorsalgia (pain in the thoracic spine)
  - Hyperkyphosis/round back of the thoracic spine

**Contra-indications**

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) in the areas of the body to be treated
- Impaired circulation or lymphatic soft tissue swelling
- Neurogenic sensory and skin trophic disorders in the body area to be treated (sensory disorders with and without skin damage)

**Side effects**

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

**Intended patient target group**

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

**Putting it on and use**

- We recommend wearing a thin, close-fitting shirt under the product.
- Open the Velcro fastener and put your arms through the arm loops of the brace **➊**, as if putting on a backpack.
- Pull both straps forward evenly with moderate tension **➋**. Adjust the tension of the arm loops to suit your correction needs.
- Fasten the straps below your chest **➌**.
- The back section should lie flat in the middle of your back between your shoulder **➍**.
- To remove the brace, open the Velcro fastener.

**Important information** ⚠

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners properly. Damage caused by improper closingof the Velcro fasteners does not constitute a reason for complaint.
- If a significant deterioration in health occurs during the use of the medical device, we ask you to report this serious incident to your specialist dealer, your doctor or us as the manufacturer, as well as to the responsible authority. You can find our contact information in these instructions for use.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- With products involving direct skin contact with an elastomer (e.g. a silicone pad), blistering may develop on the skin if the product is worn too tightly or if it is moved over the skin with high amounts of friction. In this case, we recommend stopping use of the product. Sweat can develop underneath the silicone, which in rare cases can trigger a reaction to your own sweat with corresponding allergic skin changes. SPORLASTIC is not aware of any allergic reactions being caused by the material. If you have any problems, please talk to your doctor or the supplier of the aid
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

⚠ Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

⚠ SPORLASTIC GmbH & Co. KG is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

**Care**

We recommend washing SP POSTURE gently in lukewarm water (30 °C) with mild detergent and air drying. Do not dry the product in the tumble dryer, on the heating or in the microwave.

Close the Velcro fastener when washing to ensure that it remains functional for longer and to prevent damage to other items of clothing.

**Transport and storage conditions** ✋

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

**Quality management system**

All products from SPORLASTIC GmbH & Co. KG are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

**Materials used**

Polyamid, polyurethan, polyester, elastan

⚠ An online version of the instructions for use can be found on our website at [www.sporlastic.de](http://www.sporlastic.de).

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