SWEDE-O-UNIVERSAL X8

SPORLASTIC

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INSTRUCTIONS FOR USE GB

Dear Customer,

we would like to ask you to follow these Instructions for Use with care. If you should have any questions, please consult the doctor treating you or your nearest stockist, or contact us directly.

FUNCTION

The SWEDE-O-UNIVERSAL X8 serves to stabilise the ankle joint and to prevent injury to the ankle joint during sport.

The inner-shoe construction of the SWEDE-O-UNIVERSAL X8 encloses the ankle joint. The figure eight straps provide the joint with additional stability and protect the vital ligaments of the foot. An expanding panel at the Achilles' tendon safeguards from irritation due to pressure. The heel remains free 1.

Flexible synthetic thermoplastic material reinforcements 2 give additional side stability in extreme strain. At the same time, the foot has room to move, allowing plantar flexion or dorsal extension.

INDICATIONS

- Chronic ankle joint instability
- Primary arthroses of the ankle joint
- Secondary arthroses of the ankle joint
- Prevention of recurrence of ligament rupture

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far no consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

To date, there have been no reports of serious general side effects with proper use and fitting.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restrictive, consistently shaped body fitting.

FITTING

First, thread the lace through each of the eyelets of the orthopaedic brace.

Tighten the ankle joint brace and tie the laces firmly.

TIP: Check the gap between the eyelets. If the gap is wider than 1 1/2" (4 cm), the brace is too small. If the gap is narrower than 1/2" (1 cm), the brace is too large.

- 2 Depending on stability requirements both support inlays can be inserted together into the pockets provided or each can be inserted on just one side of the ankle.
- 3 Place the foot flat on the floor and first of all draw the strap with the sewn Velcro band diagonally across the middle of the foot.
- 4, 5 Fold the strap over the ankle joint, around the lower leg, and attach the end to the strap. Apply the

second strap the same as the first strap, but in the opposite direction. The straps should be applied firmly but be comfortable to wear.

IMPORTANT

The orthotic device should always be worn over stockings.

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in con junction with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All SPORLASTIC GmbH products undergo product testing as part of our Quality Management System.

However, should you have any complaints about our product, please get in touch with your stockist or directly with us.

CARE

We recommend washing SWEDE-O-UNIVERSAL X8 gently in lukewarm water with a mild detergent and air-drying. Do not place on a radiator.

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Wszystkie wyroby SPORLASTIC GmbH podlegają kontroli w ramach naszego systemu zarządzania jakością.