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INSTRUCTIONS FOR USE EN

INTENDED USE

VERTEBRADYN® X.STRONG relieves the lumbar spine through de-lordosis, trunk pre-tilt and pelvic tilt back. For mobilisation, the brace can be gradually disengaged.

INDICATIONS

- Lumboischialgia
- Lumbar syndrome (radicular, pseudoradicular)
- Spondylolisthesis
- Lumbar deformity with hypermobility (in facet syndrome/arthritis)
- Condition after WS-OP and BS-OP with large prolapse
- Lumbar instability
- Spinal stenosis
- Vertebral fractures

CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage).

SIDE-EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

PUTTING IT ON AND USE

- We recommend wearing a thin, tight-fitting T-shirt under the product.
- Position the back section centrally over the spine. The orange-coloured lower brace edge closes over the sacrum. The back shell lies positively in the waist with the waist retraction. Place your hands in the hand loops at the front **1**.
- Guide the brace around your body and position the abdominal plate centrally at the front **2**.
- Close VERTEBRADYN® X.STRONG under the necessary tension. Make sure that the closure plates lie positively on top of each other **3**.
- Velcro the two lower tension elements under even tension over the abdomen and then the upper tension elements **4**. VERTEBRADYN® X.STRONG should be applied positively with high compression. This can provide significant relief for the lumbar spine. When fitting with high compression, make sure that it is comfortable to wear.
- The lower front end of VERTEBRADYN® X.STRONG should be flush and parallel to the thigh when seated and should not cause pressure in the groin **5**.

NOTES FOR THE ORTHOPAEDIC TECHNICIAN

- The back shell and abdominal plate are made of thermoplastic material and can be individually adapted.
- The spring steel rods can also be individually shaped.
- For increasing mobilisation, the brace can be gradually removed after consultation with the attending physician **6, 7**.
- To do this, open the Velcro fasteners between the knitted and plastic parts.
- When the abdominal plate is removed, VERTEBRADYN® X.STRONG assumes the function of a bridging brace.
- When the abdominal plate and back shell are removed, VERTEBRADYN® X.STRONG becomes a lumbar support brace.

ACCESSORIES

- The BAUCHPLATTE FÜR RÜCKENORTHESSEN can be ordered as an option for the product. It can surround prominent abdominal structures over a large area and thus support the weight of the abdomen (Art.no. 17428).
- Optionally, the VERLÄNGERUNGSTEIL RÜCKEN can be ordered with the product. It is attached ventrally to the closure plates and serves as a closure extension for larger circumferences (Art.no. 17400).

⚠ IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.

- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- Product may contain metallic objects. We recommend not putting them in contact with electromagnetic interference.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

CARE

We recommend that the VERTEBRADYN® X.STRONG knitted part and shell covers be washed gently in hand-warm water (30°C) with a mild detergent and air-dried.

Do not dry the product in the tumble dryer, on the heating or in the microwave. The plastic shells can be cleaned with a damp cloth.

Any tension in the moulded knit that decreases over the duration of wear is restored by washing. We therefore recommend washing the brace more frequently.

TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

MATERIAL COMPOSITION

Knitted fabric: 67% polyamide, 33% elastane

Belly panel and back shell: LDPE

Cover belly panel: cotton (80%) with polyester, polyamide, spandex

Back shell cover: 100% polyester.

An online version of the instructions for use can be found on our website at www.sporlastic.de.

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